

UNIVERSAL URINE SPECIMEN CONTAINER

for the safe and convenient processing of urine specimens

UC



* Flip-cap allows easy access for bacteriological sampling

* Luer adapter allows direct specimen collection via vacuum pressure



PROCESSING URINE SPECIMENS WITH THE NOVAMED UNIVERSAL

URINE SPECIMEN CONTAINER

1. Nurse gives the patient a Universal Urine Specimen Container (UUSC)* and cautions patient not to remove the cap label to protect against needlestick from the "sharp" contained in the integrated sampling device.
2. Patient is instructed to give the urine specimen to the nurse immediately after collection.
3. Use of single use disposable gloves during sampling is highly recommended.
4. For transfer of the specimen into an evacuated tube:
 - a. Place UUSC upright on clean, flat surface. Container may be tipped at an angle if specimen volume is limited.
 - b. Peel off label on cap to expose sampling ports.
 - c. Place evacuated tube into the open circular cavity on cap, stopper down. Advance the tube over puncture point to pierce stopper. For a detailed explanation refer to instructions provided by the manufacturer of the evacuated tube.
 - d. Hold tube in position until transfer is completed.
 - e. Remove tube from the UUSC and process according to standard laboratory procedures.
5. For bacteriological sampling:
 - a. Open sampling port by lifting the flip-cap.
 - b. Sample specimen with sterile bacteriological loop or with a urine culture device (Diaslide, DipStreak, dipslide).
 - c. After sampling, close flip-cap and discard UUCS into hazardous waste container

US Patent 9,295,453 B2

Universal Urine Specimen Container
Capacity - 120 ml

Catalog no. UC-120
Available in packages of 250 containers

Novamed North America
484 East Carmel Drive, Suite #282
Carmel, IN 46032
(866) 668-NOVA (866-668-6682)
customerservice@novamed-northamerica.com
www.novamed-northamerica.com