



### RESMED VPAP ADAPT



To check the hours on your machine:

Press and hold the **Green** and **Red Buttons** at the same time

Your screen will display **Efficacy Data**

Press the down arrow

Your screen will display **Usage Data**; press the **Green Button** to view

Your screen will display:

\_\_\_\_\_ Used

\_\_\_ / \_\_\_ Days, write this information down

Press down arrow

Your screen will display **Usage Hours**, will default to day

Must change to month by pressing the **Green Button 2 times**, write down the number indicated

To exit hit the right **Red Button** two times