



IAEP, Inc. is happy to announce that we are offering DE Hoof Taps for use in the treatment of common hoof deformities. IAEP has entered into a distributor agreement with DE Hoof Tapes with the goal of making this product readily available to the hoof care provider.

Now available for purchase on our website. Follow the link provided at the bottom of this page.

Case study: DE Hoof Taps

Horse: 23 year old Paso Fino Gelding

Horse has a history of chronic inflammation and osteoarthritis, which often resulted in development of imbalances within a 4 week trimming cycle.

Method: Following trimming to the axis plane hoof taps were placed at the quarters as shown.

On the fore feet a lateral tap was used on the left fore and a medial tap on the right fore.

Both medial and lateral taps were applied to both hind hooves.

Results: Shown as 5 weeks post trim

The fore feet show that improved symmetry resulted from the application of a single hoof tap.

The right fore shows that the medial hoof (medial tap) presents better symmetry than the lateral side (no tap).

The left fore show that the lateral hoof (lateral tap) presents better symmetry than medial side (no tap).

The hind feet show that symmetry and balance was maintained for the trimming cycle with the application of both medial and lateral hoof taps.

Right fore



Left fore



Right hind



Left hind



