



diagnosys
leading the wave




Envoy
**Advanced PERG
and VEP**

Desktop Envoy™


The first advanced monitor system, custom engineered by Diagnosys, that has no luminance artifact enabling precise ISCEV standard compliant PERG and VEP testing

E³ console paired with an *Envoy* monitor, patient chin rest and 32-bit amplifier create a compact system

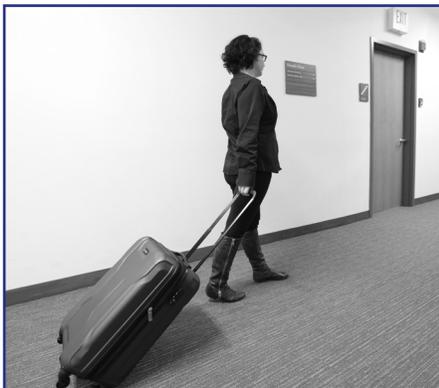

Portable System

The desktop *Envoy* system can easily be transported from one office location to another within minutes

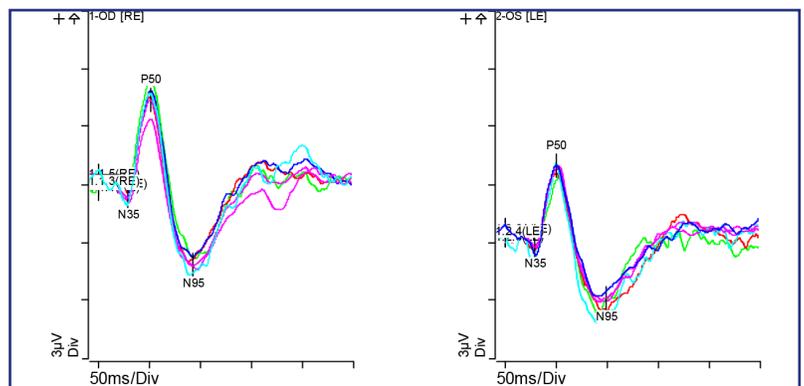

Performance Leader

Unmatched repeatability:
PERG responses from 6 different tests on the same patient

Cart-Based Envoy


Our fully integrated cart-based systems enable compact functionality in a mobile system for use throughout your facility

Monocular monitor:
Ultra high luminance and contrast monitor that is only 4"x3" in size



Profile with Envoy features:

- Operating Room arm
- Patient set up in nearby chair
- Fully integrated with the system

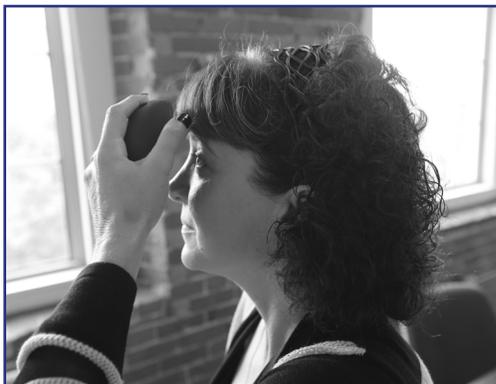

Handheld Application

The *Envoy* stimulator disconnects from its stand enabling ultimate flexibility for patient testing, including pediatric



32-Bit Amplifier

Unprecedented 1 nano Volt resolution and illuminated receptacles guide lead insertion and electrode contact quality

Macula and Ganglion Cell objective, functional testing

PERG Applications¹

- Detect & monitor dysfunction of retinal ganglion cells caused by conditions such as:
 - Glaucoma
 - Optic neuropathies
 - Primary ganglion cell diseases

VEP Applications^{2, 3}

- Detect & monitor dysfunction of optic nerve & disc function caused by conditions such as:
 - Optic nerve atrophy, including glaucomatous
 - Ischemic optic neuropathy
 - Papilledema



Envoy shown mounted on its table stand

Features

Advanced performance

- Very high luminance
- Micro-second class response times
- High contrast black and white monitor

Easy to use

- Monocular and ultra compact
- Setup in minutes & portable
- Desktop, extendable arm for cart, handheld

Industry leading protocols

- Pattern Electroretinography (PERG), ISCEV standard
- Pattern Visual Evoked Potential (VEP), ISCEV standard
- Custom PERG, including horizontal bars and larger field size
- Custom Pattern VEP, including horizontal bars and larger field size
- Onset VEP

Specifications

Display type: OLED SVGA 800x600 pixel array	Calibrated luminance: >200 cd/m ²
Contrast ratio: >300:1	Monochromatic black and white
Viewing area: approx. 12.5 x 9mm	Viewing angle: 40 degrees
Response time: <1 ms	Checks, bars, gratings as standard with 1 pixel resolution
Envoy Monitor Weight: <1 lb (0.5 kg)	Dimensions: 3.9 x 2.8 x 1.8 inches (98x70x45 mm)
Multiple spatial frequencies	

Envoy ordering information

Envoy configuration

Model number: D352

Configuration: select 1 option per field below

D352	-	
T	Table stand and chinrest	
W	Table stand and no chinrest	
A	Extendable arm for cart	
H	No table stand, no arm	

Available on the following systems:

- Diagnosis *E³* desktop systems
 - Options "T", "W", and "H"
- Diagnosis *Profile* cart-based systems
 - Options "W", "A", and "H"

References:

1. ISCEV standard for clinical pattern electroretinography (PERG): 2012 update; Bach et. al.; Doc Ophthalmol (2013) 126:1-7
2. ISCEV standard for clinical visual evoked potentials (2009 update); Odom et. al.; Doc Ophthalmol (2010) 120:111-119
3. U.S. Centers for Medicare & Medicaid Services (CMS) ICD billing codes

Application overview: Glaucoma


Research and clinical practice, including the International Society for Clinical Electrophysiology of Vision (ISCEV)^{1,2,4} have found that PERG and VEP tests may detect early optic nerve and retinal damage, including Glaucoma. PERG and VEP results using a Diagnosys system on a range of patients.

Description

- **Normal patient**
- Well formed PERG and VEP
- Large P50 and N95 responses
- N95/P50 >1.1
- VEP P100 peak near 100 ms


Summary

PERG	
P50 Amp (uv)	7.5
N95 Amp (uv)	9.1
N95/P50	1.2
VEP	
P100 peak (ms)	101

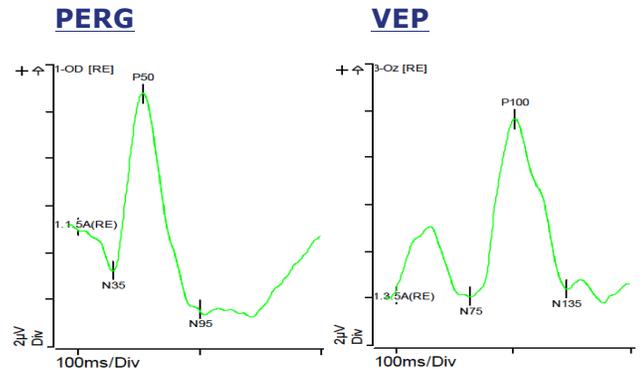


- **Abnormal patient**
- Poorly formed PERG
- Small N95 response
- N95/P50 <<1.1
- VEP P100 peak near 100 ms

PERG	
P50 Amp (uv)	4.3
N95 Amp (uv)	0.2
N95/P50	0.05
VEP	
P100 peak (ms)	102

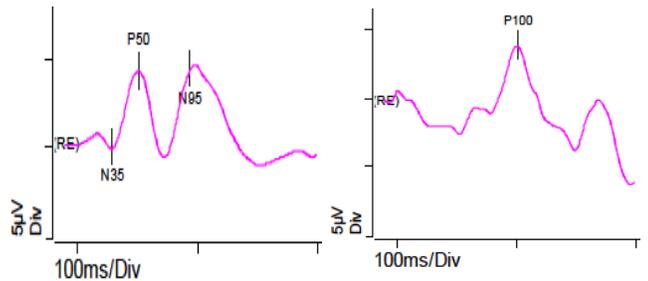

- **Highly abnormal patient**
- Poorly formed PERG
- Small N95 response
- N95/P50 <<1.1
- VEP P100 peak >> 100 ms

PERG	
P50 Amp (uv)	4.1
N95 Amp (uv)	0.1
N95/P50	0.02
VEP	
P100 peak (ms)	122

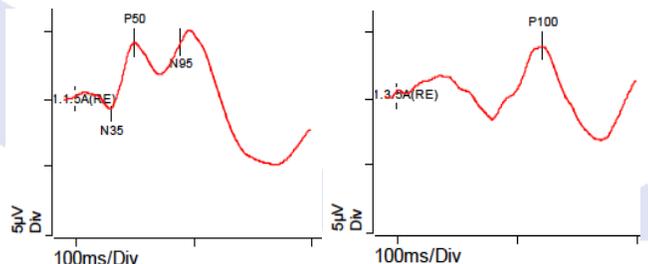

Additional references on PERG, VEP and Glaucoma:

4. ISCEV: A Guide to Procedures; Standards, Recommendations and Guidelines
5. Heckenlively, JR, et. al.: Principles and Practice of Clinical Electrophysiology of Vision; 2006; MIT Press
6. Bach M, et. al.: Check-size specific changes of PERG in patients with early open-angle glaucoma. *Doc Ophthalmol* 1987; 315-322
7. Trick GL, et. al.: PRERP abnormalities in ocular hypertension: correlation with glaucoma risk factors. *Curr Eye Res* 1988; 7:201-206
8. Seiple W, et. al.: The PERG in optic nerve disease. *Ophthalmology* 1983; 90:1127
9. Markoff JI, et. al.: Pattern reversal electroretinogram in glaucoma, glaucoma suspects and normal. *Invest Ophthalmol Vis Sci* 1983; 24 (suppl):102
10. Quigley HA, et. al.: Chronic glaucoma selectively damages large optic nerve fibers; *Invest Ophthalmol Vis Sci* 1987; 28:913-920
11. Howe, JW, et. al.: Simultaneous recoring of PERG and VEP in a group of patients with chronic glaucoma. *Doc Ophthalmol Proc Ser* 1984; 40:101-107
12. Bobak P, et. al.: PERG and VEP in glaucoma and multiple sclerosis. *Am J Ophthalmol* 1983; 96:72
13. Korth M, et. al.: PERG in normal glaucomatous eyes. *Invest Ophthalmol Vis Sci* 1987; 28 (suppl): 129
14. Weinstein GW, et. al.: The PERG in ocular hypertension and glaucoma. *Arch Ophthalmol* 1988; 106:923-928
15. Drum B, Quigley HA, et. al.: PERG and optic nerve histology in monkeys with unilateral laser induced glaucoma. *Invest Ophthalmol Vis Sci* 1986; 27 (suppl): 40

www.diagnosysllc.com


US: Diagnosys LLC USA • 175 Cabot Street, Suite 500 • Lowell, MA 01854 USA • Phone: 978.458.1600 • Fax: 978.458.1755 • Email: sales@diagnosysllc.com
UK: Diagnosys LTD UK • 5 Trust Court, Vision Park • Histon, Cambridge, CB24 9PW • Phone/Fax: +44 (0) 1223 520699 • Email: mail@diagnosysuk.co.uk

The information listed herein is accurate as of the date of printing, however may change at any time without notice. The contents may differ from the current status of approval of the product in your country. Please contact your local Diagnosys representative for more information © 2016 by Diagnosys, LLC. All copyrights reserved.
Doc: #14824 **Rev:** #A ECN 876 **Date:** 11/02/2016

Simplify your visual electrophysiology



Espion™ software provides detailed test reports that are concise and easy to interpret (ISCEV standard PERG & VEP tests using the *E³* or *Profile* system).

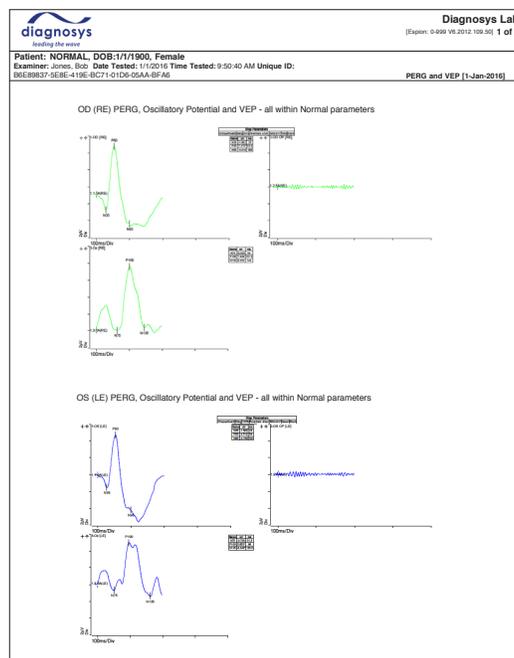

The industry leader.

Decades of experience, thousands of installations.

- Most experienced customer support team in the industry
- Hundreds of peer-reviewed papers published using our systems
- Extensive world-wide network of leading experts operating Diagnosys systems

Ordering information:

Model #: D352

Available on the following systems or as an upgrade to them:

- Diagnosys *E³* desktop systems
- Diagnosys *Profile* cart-based systems



**If you would like more information on Diagnosys products,
please contact us at either our US or UK locations.**

US: Diagnosys LLC USA • 175 Cabot Street, Suite 500 • Lowell, MA 01854 USA • Phone: 978.458.1600 • Fax: 978.458.1755 • Email: sales@diagnosysllc.com
UK: Diagnosys LTD UK • 5 Trust Court, Vision Park • Histon, Cambridge, CB24 9PW • Phone/Fax: +44 (0) 1223 520699 • Email: mail@diagnosysuk.co.uk

Doc #: 14824 **Rev:** A **ECN #:** 876 **Date:** 11/02/2016