CAREERS IN MEDICAL LABORATORY SCIENCE **OCCULT BLOOD LAB**

Follow along with the presentation to answer the questions below:

I.	The slide shows an accident scene. Once the paramedics take the patient to the hospital, name 2 medical personnel that will help care for the patient:							
	1)	2)						
II.	Healthcare careers, such as in laboratory sciences, require attention must be given to detail,, neatness, clea							
	orderliness, because their actions involve human life & well being.							
III.	III. The Hemoccult test is a screening test used for the detection of	·						
IV.	IV. Where are blood cells made? In the							
V.	V. Name one food that patient's must avoid prior to performing a Hem	occult test.						
IV.	IV. AS DIRECTED, perform a Hemoccult test & document patient and q	uality control results:						
	Materials needed: gloves, paper towel, blue/black pen, Hemoccult	card, wooden stick, timer Items to share: Patient sample, Hemoccult developer						
	a. Prepare your workstation with test materials. Obtain patient sam	ple. With gloves on, fill in the patient information on the front cover of slide						
	b. Open the flap (with the patient information area toward you) & processing the process of the	roceed:						
	 Collect a small amount of stool specimen using wooden 	stick. Apply a very thin smear inside entire box A.						
	 Reuse applicator (use other side) to obtain another sam 	ple from a different part of the stool and apply to Box B.						
	 Close cover and set timer for 4 minutes. 							
	c. Turn over slide and CAREFULLY remove perforated section to	develop test.						
	d Apply 2 drops of Hemoccult Developer over each smear. Interpr	et results within 60 seconds Positive = Blue: Negative = No Color change Record result						

- Apply 2 drops of Hemoccult Developer over each smear. Interpret results within 60 seconds. Positive = Blue; Negative = No Color change Record result.
- e. Add 1 drop between circle of Performance Monitor (ORANGE BOX). Interpret results within 10 seconds. Positive = Blue; Negative = No Color change Record result.

Careers Include: Medical Laboratory Scientist, Medical Laboratory Technician, Phlebotomist, Pathologist, Pathologists' Assistant, Clinical Chemist, Histotechnologist, Cytotechnologist, Technologist in Cytogenetics, Technologist in Molecular Biology, Forensic Scientist, Crime Scene Investigator, & Others!

OCCULT BLOOD LOG SHEET

	Date	Time	PATIENT NAME (last name, first name)	DOB	Tech	MD	Slide Lot # & Exp. Date	Developer Lot # & Exp. Date	Occult Blood RESULT	Positive Control Result	Negative Control Result
i											