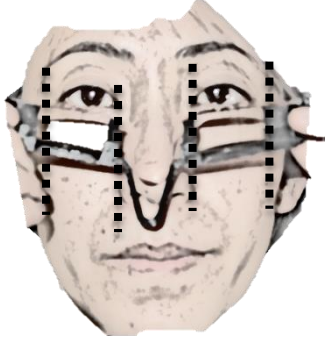
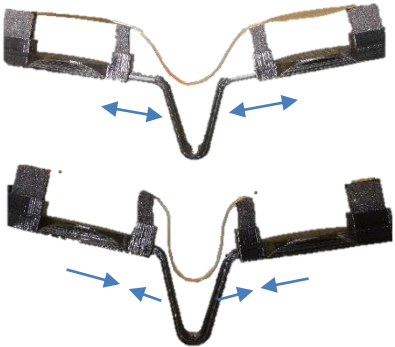
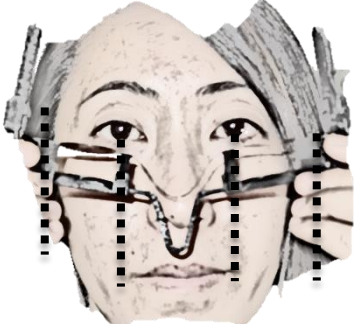



Instructions on how to use the Heated Eye Pad
Read ALL Safety Precautions before use.

	<p>Clean the device before each use by wiping with a rinse free eyelid wipe.</p> <p>Look in the mirror. Adjust the nose part of the Heated Eye Pad to fit your face and eyes. This is correct.</p> <p>The round section of the Heated Eye Pad should line up with your eyes, as shown in between the dashed lines.</p>
	<p>Adjust the nose part of the Heated Eye Pad to fit your face and eyes.</p>
	<p>As you adjust the frame, recheck alignment to the eye. This is incorrect</p>
	<ol style="list-style-type: none"> 1. Setup a timer (Watch, Clock, Phone, Etc.) if you wish to time your session. Users typically apply the device 5-10 minutes, twice per day. 2. Plug the heater into the power supply furnished by Digital Heat. Do not use other power supplies. 3. Gently place the Heated Eye Pad on your CLOSED eyelids and position the frame ends behind ears. 4. Adjust the tightness of the frame for comfort, and place the electrical cord behind an ear. 5. DO NOT apply excessive pressure on your eyelids by over tightening the frame. 6. Repeat steps 1 through 6 as needed for comfort. 7. If timed, after timer has expired remove the Heated Eye Pad from your eyelids and face.