

IntelliPAP BILEVEL

ENTER THE CLINICIAN SET-UP MODE

1. With the device plugged in and powered off press and hold the DOWN (▼) and DELAY (▲) buttons.
2. While holding down the two buttons press the ON/OFF button. The display will show “Clinical Menu”. You can now access the Clinical Setup Menu.

Use the LEFT button and RIGHT button to navigate through the menu. Use the UP button and DOWN button to adjust the settings.

Clinical Menu Order (BiLevel S Mode)

1. Language
2. Operating Mode – Select BiLevel S
3. IPAP – Set to IPAP prescribed by Doctor (default 15.0 cmH₂O)
4. EPAP – Set to EPAP prescribed by Doctor (default 8.0 cm H₂O)
5. Delay Time – Select time to stay at delay pressure. The delay is patient activated. (Default 20 min)
6. Delay Pressure – If prescribed set to a level comfortable for patient to fall asleep. Pressure starts at set level and rises to the prescribed IPAP/EPAP level over the delay time. (Default 4.0 cmH₂O)
7. IPAP Rounding – Pressure waveform profile. Range 1-5. (default 3)
8. EPAP Rounding - Pressure waveform profile. Range 1-5. (default 3)
9. Inspiratory Trigger Sensitivity – Adjust to control inhale trigger sensitivity 1-10. 1=most, 10= least sensitive. (default 5)
10. Expiratory Trigger Sensitivity – Adjust to control exhale trigger sensitivity 1-10. 1=most, 10= least sensitive. (default 5)
11. Pressure Unit – Choose how pressure is displayed. (default cmH₂O)
12. Tubing Length – Choose tubing length for max pressure accuracy (default 1.8 meters, 6 feet)
13. Usage Threshold – Smart Code tracks day of minimum usage. 4 or 5 hours (default 4 hours)
14. Compliance Meter – Displays cumulative hours of breathing time. To clear meter press ▼ button for 8 seconds.
15. Enable Menu - ▼
16. Lock Menu - ▼