




FISHER & PAYKEL ICON NOVO CPAP



To check the hours on your machine:

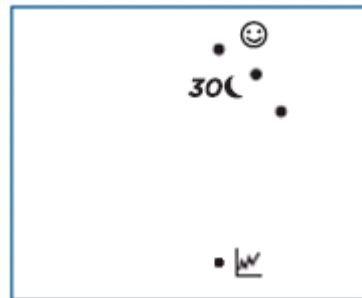

Turn the SmartDial to the Sleep Data setting and press.

/

Screen will display as above:

Press the SmartDial to enter the menu with the smiley face

Turn the SmartDial to scroll through each of the following statistics:

- Total Compliant Hours .
- Average Compliant Hours over the last 30 nights.
- Percentage of nights used greater than 4 hours over the last 30 nights.
- The number of nights used since therapy commenced.
- Exit the Sleep Data setting.

Please complete the following: