


Fascial Closure Kit

for visualized full-thickness closure

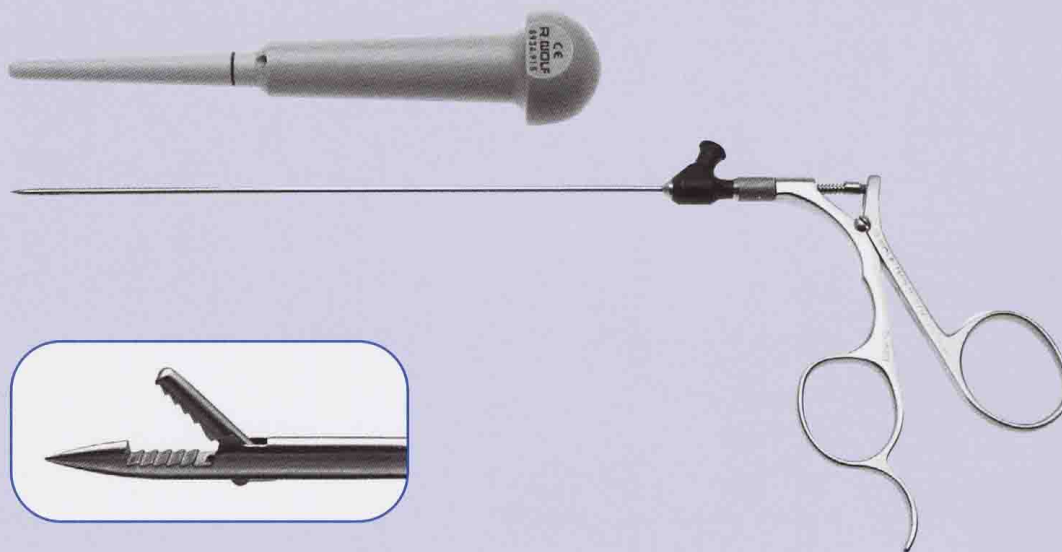

The extended tip of the fascial closure guide allows for easy insertion into the trocar defect. The fascial closure guide preserves the pneumoperitoneum after the trocar cannula has been removed.

A suture is grasped with the fascial closure forceps and inserted through the hole in the fascial closure guide, passing through the muscle, fascia and peritoneal layers. The suture is released inside the peritoneal cavity. To complete the stitch on the other side, the device is kept in place and turned 180°. The grasper is reinserted and the intraperitoneal end of the suture is grasped and withdrawn from the peritoneal cavity as the fascial closure guide is removed. The sutures are tied under direct visual control.

Additional applications include: Ligation of abdominal wall bleeding and for fixation of mesh.

ORDERING INFORMATION:

Fascial Closure Kit:8756.205

Closure guide 10-12 mm (8934.915), closure guide 12-15 mm (8943.915), closure guide 15-20 mm (8945.915), needle-tipped grasper (8756.201) and instrument tray for storage and sterilization (8585.091M)

Full-Thickness Closure

