DEALER: These instructions MUST be given to the user of this product.

NOTE: Check ALL parts for shipping damage. If shipping damage is noted, DO NOT use. Contact carrier/dealer for further instruction.

⚠ WARNING

- Do not use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as owner's manuals, service manuals or instruction sheets supplied with this product or optional equipment. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.
- Check to make sure that the rubber tips on the leg extensions are not ripped, worn or missing. If necessary, replace all missing or damaged parts **BEFORE** using the Toilet Safety Frame.
- Users with limited physical capabilities should be supervised or assisted when using the Toilet Safety Frame.
- The Toilet Safety Frame is designed to provide support, increase stability and assist the end-user. The Toilet Safety Frame is **NOT** designed to support the total weight of an individual.
- **ALWAYS** observe the weight limit on the labeling of your product. Check that all labels are present and legible. Replace if necessary.

⚠ INSTALLATION WARNINGS

- Ensure that **ALL** hardware is tight at **ALL** times.
- Ensure snap buttons are fully protruding through the same adjustment hole before using. This ensures that the adjustable legs and arm supports are securely locked in position and engaged in the same adjustment hole for an even height and width adjustment.

⚠ ACCESSORIES WARNING

- Invacare products are specifically designed and manufactured for use in conjunction with Invacare accessories. Accessories designed by other manufacturers have not been tested by Invacare and are not recommended for use with Invacare products.

Installing the Toilet Safety Frame

NOTE: For this procedure, refer to FIGURE 1.

1. Remove the toilet seat and lid hardware from the toilet bowl.
2. Remove the toilet seat and lid.
3. Align the holes on the mounting bracket with holes on the toilet bowl.
4. Put the toilet seat and lid in place on top of the mounting bracket.
5. Attach toilet seat, lid and mounting bracket to the toilet. Tighten securely.

⚠ WARNING

Use the inner adjustment holes when attaching the arm supports to increase stability and balance, otherwise - injury or damage may occur.

6. On one arm support, press the snap button and insert the arm support into the mounting bracket. Slide the snap button to the inner adjustment hole.

NOTE: There will be an audible "click".

NOTE: Ensure that the snap button fully protrudes through the hole in the mounting bracket.

7. Repeat STEP 6 to install the opposite arm support.


8. Press the snap button and slide the leg extension up or down to the desired hole.

NOTE: Ensure that the snap button fully protrudes through the hole in the leg extension.

NOTE: There will be an audible "click".

9. Repeat STEP 8 to install or adjust the opposite leg extension.

NOTE: Make sure that the toilet safety frame sits level and is secure before use.

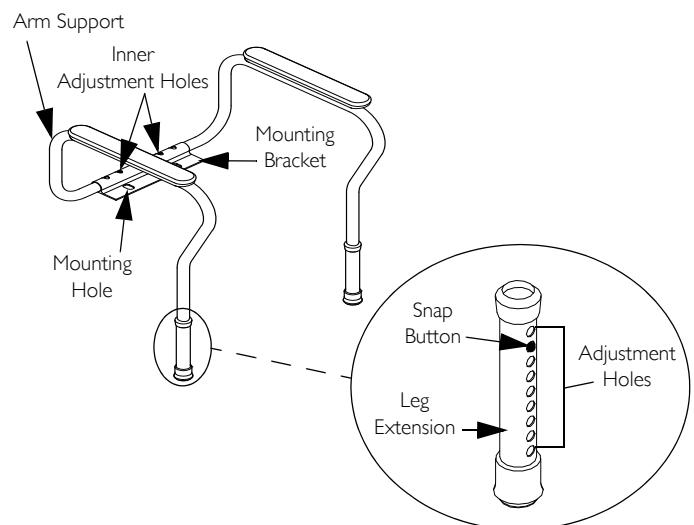DETAIL "A" - TOILET SAFETY FRAME SHOWN IN PLACE

FIGURE 1 Installing the Toilet Safety Frame

For more information regarding Invacare products, parts, and services, please visit www.invacare.com

LIMITED WARRANTY

PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to the original purchaser/user of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants all this product to be free from defects in materials and workmanship for three years for the original purchaser. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option, with refurbished or new parts. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. any such product. Product repairs shall not extend this warranty - coverage for repaired product shall end when this limited warranty terminates. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address on the back cover. Provide dealer's name, address, model number, and the date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number.

Invacare Corporation will issue a return authorization. The defective unit or parts must be returned for warranty inspection using the serial number, when applicable, as identification within thirty (30) days of return authorization date. **DO NOT** return products to our factory without our prior consent. **C.O.D.** shipments will be refused; please prepay shipping charges.

LIMITATIONS AND EXCLUSIONS: THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE ENCLOSED INSTRUCTIONS. IN ADDITION, THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE; OR PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT INCLUDING, BUT NOT LIMITED TO: MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS; PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE; PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL; PRODUCTS REPAIRED BY ANYONE OTHER THAN AN INVACARE DEALER, SUCH EVALUATION SHALL BE SOLELY DETERMINED BY INVACARE.

THE FOREGOING WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE.

IT SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESSED WARRANTY PROVIDED HEREIN AND THE REMEDY FOR VIOLATIONS OF ANY IMPLIED WARRANTY SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.

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All rights reserved. Trademarks are identified by the symbols ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.
© 2009 Invacare Corporation