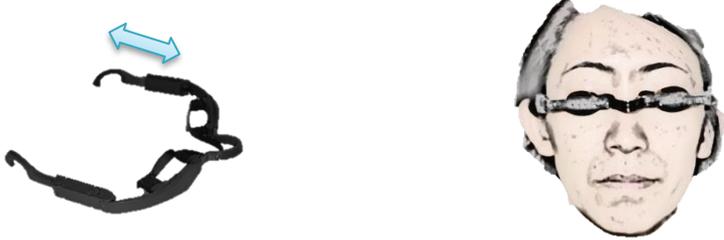


# SCORPION 1-TOUCH EYELID HEATER

[www.heatedeyepad.com](http://www.heatedeyepad.com)  
(512) 517-6649

by **DIGITAL HEAT**  
5626 S. Captain Kidd Ct., Unit B  
Tempe, AZ 85283

Read ALL safety precautions before use.

<p><b>1. Clean</b></p>  <p style="text-align: center;"><b>CORRECT</b>                      <b>INCORRECT</b></p>	<p>Clean the device before each use by wiping with a rinse free eyelid wipe or alcohol wipe.</p> <p>Never pull the heater upward, as this will damage or tear the foam away.</p>						
<p><b>2. Adjust</b></p>  <p style="text-align: center;"><b>CORRECT</b>                      <b>INCORRECT</b></p>	<p>Hold nose part of the eyelid heater to fit the eyes.</p> <p>Device should align with the canthus or corners of the eyes.</p> <p>Adjust the frame as needed for alignment to the eyes.</p>						
	<p>Adjust the temple to correct the length over the ears.</p> <p>Gently place onto CLOSED eyelids and position the frame ends behind ears.</p>						
<p><b>3. Plug In</b></p>	<p>Plug the Digital Heat supplied Power Bank into the device.</p>						
<p><b>4. Set the Temperature</b></p> 	<p>Press button once for HIGH, Press twice for MEDIUM, and Press three times for LOW.</p> <p>Pause and the device opts for TIME.</p>						
<p>TIME modes:</p> <table border="0"> <tr> <td>HIGH</td> <td>10 minutes maximum</td> </tr> <tr> <td>MEDIUM</td> <td>15 minutes maximum</td> </tr> <tr> <td>LOW</td> <td>30 minutes maximum</td> </tr> </table>	HIGH	10 minutes maximum	MEDIUM	15 minutes maximum	LOW	30 minutes maximum	<p>Press Button to lower the time, each click has a sound and represents 1 minute.</p> <p>Pause and the device starts beeping upon completion.</p>
HIGH	10 minutes maximum						
MEDIUM	15 minutes maximum						
LOW	30 minutes maximum						
<p><b>5. Remove</b></p>	<p>Remove and place the eyelid heater away.</p>						

# SCORPION 1-TOUCH EYELID HEATER

[www.heatedeyepad.com](http://www.heatedeyepad.com)  
(512) 517-6649

by **DIGITAL**  **HEAT**  
5626 S. Captain Kidd Ct., Unit B  
Tempe, AZ 85283

## INDICATIONS FOR USE

The Scorpion 1-Touch ophthalmic warmer is a powered heating pad for the application of localized heat therapy. Use for treatment when the current medical community recommends the application of a warm compress to the eyelids. Such applications would include Meibomian Gland Dysfunction (MGD), Dry Eye, Blepharitis, Styte, or Chalazia.

Surface temperature of this device can reach 55 °C or 131 °F, and eyelid skin temperatures can reach 44 °C or 111 °F.

The clinical effect of elevated skin temperatures are as follows according to the National Institute of Standards and Technology: American Society for Testing and Materials, ASTM C1055, Standard Guide for Heated Systems Surface Conditions That Produce Contact Burn Injuries, 4:6, ASTM West Conshohocken, PA, 1997:

44 °C (111 °F)	Human skin temperature when pain is felt
48 °C (118 °F)	Human skin temperature causing a first degree burn injury
55 °C (131 °F)	Human skin temperature with blistering and second degree burn injury

## IMPORTANT SAFETY INSTRUCTIONS

**WARNING:** Use carefully. May cause serious burns. **Do Not** use over sensitive skin areas or in the presence of poor circulation. The unattended use of the Scorpion Compress by children or incapacitated persons may be dangerous. To reduce the risk of burns, electrical shock, fire, and accident, this product must be used in accordance with the following instructions:

- 1. Read all instructions carefully**
- 2. WARNING:** Device may cause skin irritation or burning sensation. Do not use on sensitive skin areas or in the presence of poor circulation. This product not intended for use by incapacitated individuals.
- 3. DO NOT** use pad on infants.
- 4.** This pad is not to be used on or by an invalid, a paralyzed person, a sleeping or unconscious person, a person with diabetes, or a person with poor blood circulation. Do not use eye pad on areas of sensitive skin.
- 5.** Burns may occur. Check skin under pad frequently to avoid burning and blistering.
- 6. Place pad on top of closed eyelids. Never on eyes, or eyeball.**
- 7. DO NOT** use pins or other metallic means to fasten heater eye pad in place.
- 8. DO NOT** fold, bend, crush, lie, or sit on heater eye pad to avoid damage to device.
- 9. Never** pull this pad by the supply cord and do not use the cord as a handle.
- 10. Carefully** examine before each use. Discard the pad if it shows any sign of deterioration (such as blistering or cracking).
- 11. Only Use** the power supply provided by Digital Heat.
- 12. Unplug** pad when not in use.
- 13. DO NOT** tamper or modify the materials/configuration. There are no user serviceable parts. If for any reason this pad does not function satisfactorily, contact/return to Digital Heat.
- 14. DO NOT** use this pad while wearing contact lenses or with other eye care products.
- 15. DO NOT** use this pad with any liniments, salve, ointments, liquids, or any other materials in associations with the specified usage instructions of this device.
- 16. DO NOT** use this pad while taking sensory dulling medication.
- 17. DO NOT** wraps cord tightly or around eye pad heater to avoid damage to components. Loop cord lightly for storage to avoid any damage.
- 18. Save** these instructions.

### Contact Information:

**Web:** [www.heatedeyepad.com](http://www.heatedeyepad.com)  
**Email:** [Info@digheat.com](mailto:Info@digheat.com)  
**Technical Support:** (512) 517-6649