


TITAN Series LOCTITE Instructions



1. *Prior to patient receiving the finished prosthesis, all screws should be set with LOCTITE 242 (Included with all TITAN series fingers)*
2. *Remove the joint screws with a 2 mm Allen wrench.*
3. *Apply LOCTITE 242 to threads of screws and re-insert back in to TITAN finger.*
4. *Screws should be hand tightened.*
5. *Remove 8-32 screw which connects the TITAN series finger to the mounting bracket.*
6. *Apply LOCTITE 242 to threads of the screw and re-insert back in to the TITAN series finger.*
7. *Screw should be secured in place with #2 Phillips screw driver*


TITAN FULL

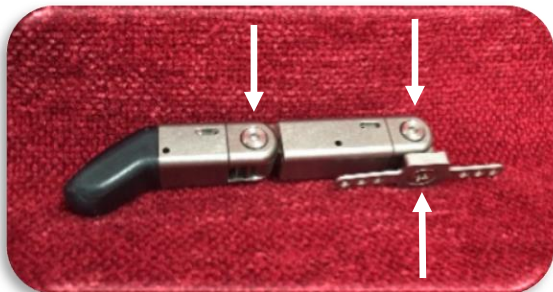

TITAN PARTIAL



TITAN THUMB

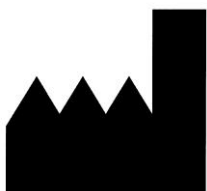

PARTIAL HAND SOLUTIONS

86 Ladyslipper Lane
Southington, CT, USA 06489
+1 (860) 538-5532

EC

REP

EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands

