
M-Finger Fabrication for 1,2,3 or 4 fingers

4/2017



1. Modify mold



2. Identify location of mounting plate base and build up the mounting plate base with plaster and smooth into mold. Make sure to check alignment of the fingers.



3. If using an M-Thumb, place the thumb mounting dummy in the desired location and screw into mold with provided screw.



4. Identify location for Gaffney joint (typically placed on ulnar side) and plaster in place and set with provided nails.



Photo Showing Gaffney on Radial side

5. Apply PVA bag over mold and laminate.



6. Trim out to desired trimlines.

7. Install BOA cable connector or preferred cable attachment method in proper location.