


EASY ADJUSTMENT

*Adjustments easier than ever
before by our latest design!!*

DON'T TRY TO MAKE ANY ADJUSTMENTS BY MEANS OF THE CABLES. THEY ARE SET UP AT THE FACTORY FOR MAXIMUM TRAVEL AND EASE OF MOVEMENT. THE EASY WAY TO ADJUST THE LOCK PIN INTO THE HOLES OF THE DISC IS BY LOOSENING THE SCREWS AND MOVE THE 3/4" DIAMETER TUBE CLOSER OR FURTHER AWAY FROM THE HOLES IN THE DISC LEAVING ABOUT 1/8" CLEARANCE BETWEEN THE LOCK PIN AND THE DISC WHEN IT IS IN THE FREEWHEELING POSITION. THERE SHOULD BE NO CLICKING SOUND AFTER ADJUSTMENT.

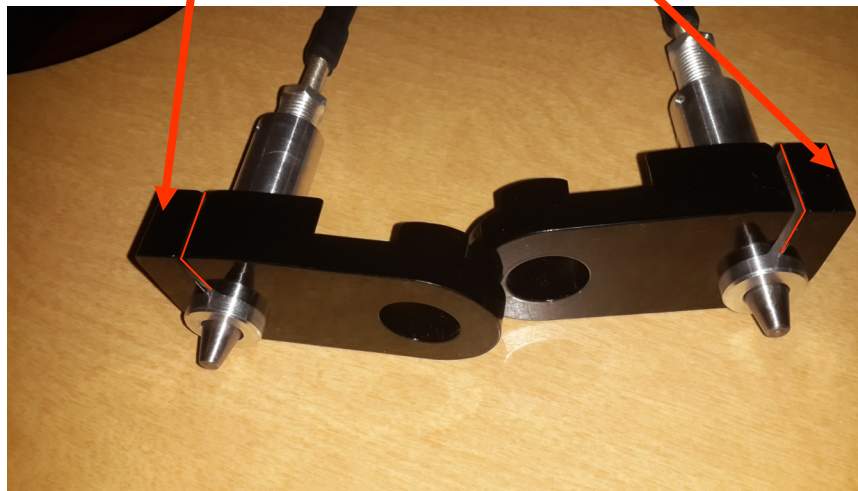

Parts for Quickie 2 Wheelchair

