Diagnostic Audiometer





# The true hybrid

- Stand alone & PC-based audiometry in one box





## Diagnostic Audiometer

# The first 'true' hybrid audiometer

The AD629 presents a new concept we like to call 'hybrid', because it combines the best of past, present and future in a compact and classic design. It combines the best features of our AD229 and brings it to a new level with modern PC integration.

### **Integrated functionality**

The AD629 has the familiar feel of a standalone audiometer but offers features usually restricted to PC platforms such as EMR integration, multiple data/patient storage features, reporting tools and integration with other database programs (NOAH, OtoAccess™).

## Familiarity + functional design

The ergonomic design of the AD629 is exceptional with an articulating, high-resolution 5.7 inch color display that presents crystal clear images for easy to read audiograms. Several display choices are available to customize the readout to your personal preference. The familiar keypad design will enable users to acclimate quickly without losing efficiency or speed.

### **Superior versatility**

The AD629 is small for portability, yet presents a high professional image in any environment. The power of the internal processor provides PC-like functionality without the PC! Upload multiple patient data into the AD629 before taking it on the road, then store test results to each specific client and download the information when you get back to the office to the database program of your choice (NOAH, OtoAccess<sup>TM</sup>).



#### **Benefits**

- Full PC-integration (true hybrid)
- · High performance and speed
- Versatility
- Extensive diagnostic test battery

### **Highlights**

- Large 5,7" adjustable high resolution graphical color display
- Internal storage for 500 patients/ 50,000 test sessions
- Integrated pre-recorded speech
- · Direct print options
- Compact and portable design (14 x 11 inch.)

# Tradition and innovation

### Tradition blended with innovation

The AD629 provides all of the traditional testing capabilities of a diagnostic audiometer but now blends additional measures and features to make your life simpler and more efficient. Built-in wavefiles for a variety of speech evaluations – including QuickSin, extended high frequency evaluation, PTA calculations and more. Hardware for efficient patient communication interaction is provided for sound room applications.

The AD629 – your dependable partner

- **MEASURE** using an intuitive and time tested user interface
- **VIEW** results on the crystal clear color display
- **STORE** data in multiple data sources
- **COUNSEL** patients
- **SHARE** data with others using reporting tools







# Diagnostic Audiometer

# The diagnostic test battery

AD629e provides all the necessary air, bone and masking features required for basic or advanced diagnostic and clinical use. The built-in high resolution color display offers dedicated test screens as well and many display and test options.

## **Tone audiometry**

Automatic CPT and PTA calculation

User defined symbol scheme

Easy access to main functions



Dual audiogram mode


## Tone audiometry test features

- Air conduction (HL, MCL & UCL)
- Bone conduction
- Pure Tone Stenger
- PTA and/or CPT calculation
- Define special audiometry symbols
- Easily switch to 1,2 or 5dB steps
- Display masking information

#### Other available tests

- · Master Hearing Aid
- SISI
- Auto HL (Modified Hughson-Westlake)
- ABLB
- Bekesy
- High Frequency to 20,000 Hz (optional)

## **Speech audiometry**

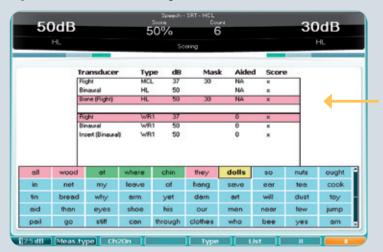

Choose between graph or table mode





Built-in media player for playing prerecorded speech material



## **Speech Audiometry Test Features**

- Live Voice, CD/MP3 or Wavefile presentations
- SRT, Word Recognition, MCL, UCL
- Recorded Word Recognition Scoring
- Binaural speech for CAPD applications
- Speech Stenger
- Built-in Wavefiles (optional)
- QuickSin (optional)



# Diagnostic Audiometer

# The true hybrid - overview

## **Large color display** 5.7" adjustable high

5.7" adjustable high resolution graphical color display

2 Channel tone and speech testing with dedicated test screens

Fully customizable

Via user friendly setups


Integrated recorded speech material

Two way patient communication

High Frequency option (20,000 Hz)

**Built-in power supply** 



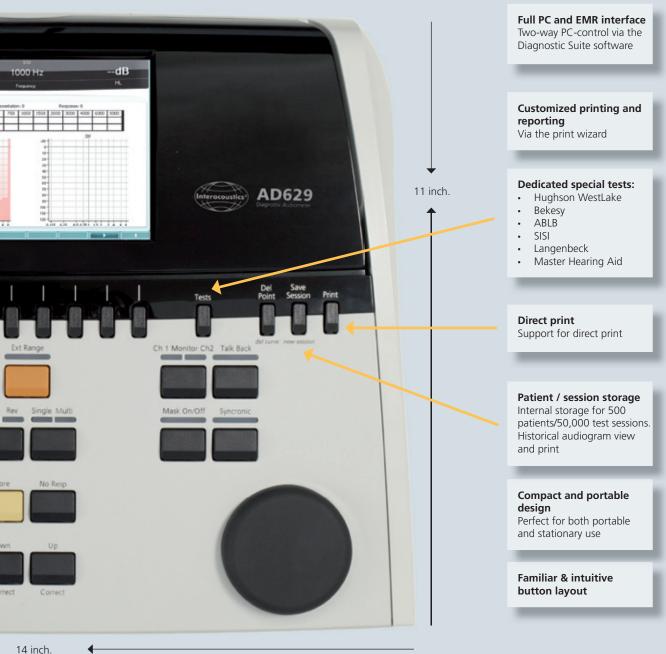



# Diagnostic Audiometer

# Printing and reporting

## Printing and reporting with 'Print Wizard'


- Unlimited customizable reports
- Combine diagnostic data (Audiometry/Impedance and more)
- EMR and 3rd Party integration via PDF, XML or GDT formats
- Merge written information from imbedded report writer

### **Data storage**


The AD629 may run independent of a database, but is designed for two way communication to merge data between NOAH and our own proprietary database called OtoAccess<sup>TM</sup>.

#### Platforms for the future

Interacoustics 'Suite Software' concept provides a dynamic core for all future instrumentation design, where instrument integration and upgrades will continue for years to come.

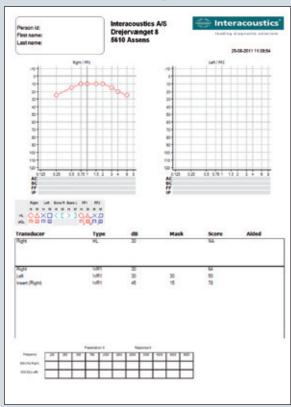

Direct printout via printer connected to the AD629

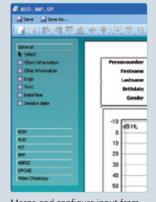

Merge and configure input from different sources in the Print Wizard



# Design your own reports

Retrieve patient demographics from OtoAccess™ and NOAH

Merge audiometric and impedance test data on a single page Interacoustics A/S
7625 Goldentriangle Dr.
Eden Prairie, MN 55344

| Figure | Figure

Insert bitmaps such as clinic logo

Individual session dates

Highly configurable symbols for quick reference

Text field for session observations and conclusions

Printout via Diagnostic Suite software and the print wizard

counseled on hearing protection.

Test Reliability: Good

Examiner: David P. Speidel, M.S. Audiologist






## Combinations & Specifications

# Popular combinations

Interacoustics provide a comprehensive software based solution for audiologic diagnostics and hearing aid fitting. Through our intuitive and easy to use software suites the AD629 can easily be combined and integrated with our other instruments.

The Interacoustics Suites integrate various measurement modules in a common yet intuitive and user friendly way to enable viewing, storing and printing of shared data.

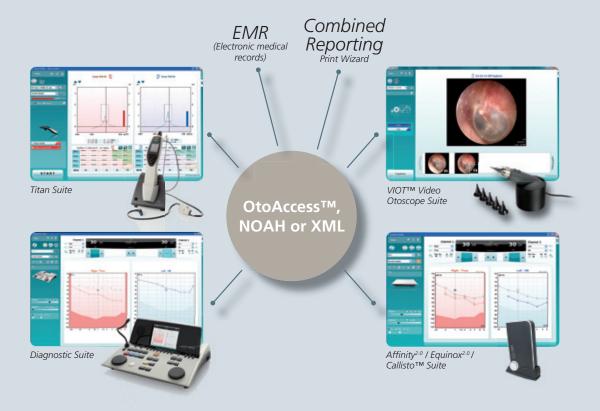

Diagnostic Suite supports all stand alone audiometers and tympanometers: AS608/AS608e, AD226, AD229b/AD229e, AD629, AC33, AC40, MT10, AT235/AT235h, AA222/AA222h.

IEC60601-1, ES60601-1, CAN/CSA-C22.2 No.60601-1  Class I, Applied parts type B. Continuous operation  Audiometer: IEC 60645 -1, ANSI S3.6, type 2. Speech: IEC 60645-2/ANSI S3.6, type B or B-E.  AC: ISO389-1, ISO389-2; BC: ISO389-3, IEC 60645-4. (Optional license)  If not activated, the AC output will be limited to 20 dB below maximum output.  Tone, Warble Tone, NB, CD1, CD2, Mic1, Mic2, wave files.  Automatic selection of narrow band noise (or white noise) for tone presentation and speech noise for speech presentation.  AC Left + Right, BC Left + Right, Insert Phone Left + Right, Insert Masking, FF1 and FF2.  TDH39/DD45 Audiometric headset, B71 Bone conductor.  Manual or reverse. Single or multiple pulses.  One hand held push button.  AC: -10 to 120 dB HL in 1, 2 or 5dB steps. BC: -10 to 80 dB in 1, 2 or 5 dB steps  125Hz to 8kHz. Optional HF up to 20 Hz.  Talk Forward and Talk Back.
Audiometer: IEC 60645 -1, ANSI S3.6, type 2. Speech: IEC 60645-2/ANSI S3.6, type B or B-E.  AC: ISO389-1, ISO389-2; BC: ISO389-3, IEC 60645-4. (Optional license)  If not activated, the AC output will be limited to 20 dB below maximum output.  Tone, Warble Tone, NB, CD1, CD2, Mic1, Mic2, wave files.  Automatic selection of narrow band noise (or white noise) for tone presentation and speech noise for speech presentation.  AC Left + Right, BC Left + Right, Insert Phone Left + Right, Insert Masking, FF1 and FF2.  TDH39/DD45 Audiometric headset, B71 Bone conductor.  Manual or reverse. Single or multiple pulses.  One hand held push button.  AC: -10 to 120 dB HL in 1, 2 or 5dB steps. BC: -10 to 80 dB in 1, 2 or 5 dB steps  125Hz to 8kHz. Optional HF up to 20 Hz.
AC: ISO389-1, ISO389-2; BC: ISO389-3, IEC 60645-4. (Optional license)  If not activated, the AC output will be limited to 20 dB below maximum output.  Tone, Warble Tone, NB, CD1, CD2, Mic1, Mic2, wave files.  Automatic selection of narrow band noise (or white noise) for tone presentation and speech noise for speech presentation.  AC Left + Right, BC Left + Right, Insert Phone Left + Right, Insert Masking, FF1 and FF2.  TDH39/DD45 Audiometric headset, B71 Bone conductor.  Manual or reverse. Single or multiple pulses.  One hand held push button.  AC: -10 to 120 dB HL in 1, 2 or 5dB steps. BC: -10 to 80 dB in 1, 2 or 5 dB steps  125Hz to 8kHz. Optional HF up to 20 Hz.
If not activated, the AC output will be limited to 20 dB below maximum output.  Tone, Warble Tone, NB, CD1, CD2, Mic1, Mic2, wave files.  Automatic selection of narrow band noise (or white noise) for tone presentation and speech noise for speech presentation.  AC Left + Right, BC Left + Right, Insert Phone Left + Right, Insert Masking, FF1 and FF2.  TDH39/DD45 Audiometric headset, B71 Bone conductor.  Manual or reverse. Single or multiple pulses.  One hand held push button.  AC: -10 to 120 dB HL in 1, 2 or 5dB steps. BC: -10 to 80 dB in 1, 2 or 5 dB steps  125Hz to 8kHz. Optional HF up to 20 Hz.
Tone, Warble Tone, NB, CD1, CD2, Mic1, Mic2, wave files.  Automatic selection of narrow band noise (or white noise) for tone presentation and speech noise for speech presentation.  AC Left + Right, BC Left + Right, Insert Phone Left + Right, Insert Masking, FF1 and FF2.  TDH39/DD45 Audiometric headset, B71 Bone conductor.  Manual or reverse. Single or multiple pulses.  One hand held push button.  AC: -10 to 120 dB HL in 1, 2 or 5dB steps. BC: -10 to 80 dB in 1, 2 or 5 dB steps  125Hz to 8kHz. Optional HF up to 20 Hz.
Automatic selection of narrow band noise (or white noise) for tone presentation and speech noise for speech presentation.  AC Left + Right, BC Left + Right, Insert Phone Left + Right, Insert Masking, FF1 and FF2.  TDH39/DD45 Audiometric headset, B71 Bone conductor.  Manual or reverse. Single or multiple pulses.  One hand held push button.  AC: -10 to 120 dB HL in 1, 2 or 5dB steps. BC: -10 to 80 dB in 1, 2 or 5 dB steps  125Hz to 8kHz. Optional HF up to 20 Hz.
noise for speech presentation.  AC Left + Right, BC Left + Right, Insert Phone Left + Right, Insert Masking, FF1 and FF2.  TDH39/DD45 Audiometric headset, B71 Bone conductor.  Manual or reverse. Single or multiple pulses.  One hand held push button.  AC: -10 to 120 dB HL in 1, 2 or 5dB steps. BC: -10 to 80 dB in 1, 2 or 5 dB steps  125Hz to 8kHz. Optional HF up to 20 Hz.
TDH39/DD45 Audiometric headset, B71 Bone conductor.  Manual or reverse. Single or multiple pulses.  One hand held push button.  AC: -10 to 120 dB HL in 1, 2 or 5dB steps. BC: -10 to 80 dB in 1, 2 or 5 dB steps  125Hz to 8kHz. Optional HF up to 20 Hz.
Manual or reverse. Single or multiple pulses.  One hand held push button.  AC: -10 to 120 dB HL in 1, 2 or 5dB steps. BC: -10 to 80 dB in 1, 2 or 5 dB steps  125Hz to 8kHz. Optional HF up to 20 Hz.
One hand held push button.  AC: -10 to 120 dB HL in 1, 2 or 5dB steps. BC: -10 to 80 dB in 1, 2 or 5 dB steps  125Hz to 8kHz. Optional HF up to 20 Hz.
AC: -10 to 120 dB HL in 1, 2 or 5dB steps. BC: -10 to 80 dB in 1, 2 or 5 dB steps 125Hz to 8kHz. Optional HF up to 20 Hz.
125Hz to 8kHz. Optional HF up to 20 Hz.
Talk Forward and Talk Back.
Output through built-in speaker or through external earphone or speaker.
Internal storage for 500 patients/50,000 test sessions.
125Hz, 250Hz, 750Hz, 1500Hz or 8kHz may freely be deselected
Auto threshold, ABLB, Stenger, Stenger Speech. Extended version (AD629e) only: SISI, Langenbeck (tone in noise), Békésy, 2 channel speech, 2 channel Master Hearing Aid.
5.7 inch high resolution color display 640x480 pixels
Back: 4 USB, 1 Ethernet, 2 Free Field (phono), Phone L/R (large jack), Insert L/R (large jack), Bone (large jack), Patient Response (large jack), Talk Back (large jack), Mic 2 (large jack), CD1 (phono). Left side: Headset (phono), Mic 1 (phono)
Supported printers: HP (PCL 3, HP PCL 5e), Epson (ESC/P2, LQ, Stylus Color), Cannon (iP100, iP90, BubbleJet), PocketJet II, 200, 3, Porti-W40, MTE Mobile Pro Spectrum, M-300, Zebra CPCL printers
Standard PC keyboard mouse and keyboard (for data entry)
Diagnostic Suite - NOAH, OtoAccess™ and XML compatible
14.4x11.6 x2.6 inches
6.3lbs
Built-in power supply, 100-240 V, max 0.5Amp
Yes
EARTone 3A Insert Phones B71 Bone conductor Monitor Headset APS3 Patient response button Goose neck microphone Power cable Operation manual CD Multilingual CE instructions for use
DD45 Audiometric headset Diagnostic Suite software OtoAccess™ database 21925 Amplivox audiocups, noise reducing headset 50250 Peltor noise reducing headset Carrying case (Standard or Trolly Style) HDA200 Audiometric headset CIR22 Insert earphone set for masking or monitoring EM400 Talk back microphone MTH400 Monitor headset MTH400M Monitor headset with boom mic. Sound field speakers

#### Interacoustics - the wise choice

With over 40 years of experience, Interacoustics is dedicated to supplying its customers with the best possible solutions for their audiologic needs. This is accomplished by maintaining a continuous dialogue with healthcare professionals working in all sectors of audiology. Our equipment meets the highest possible engineering standards and we provide design expertise that can only come from close contact with clinical practice.

#### Solutions on every scale

Designing equipment for every size of clinic in so many countries puts us in the unique position of being able to offer solutions that fit your requirements exactly. Audiometry, tympanometry, electrophysiology, hearing aid testing, balance investigation are all within our scope and can be integrated to suit your needs.

### **Designed for diagnostics efficiency**

We design equipment to make testing and interpretation easier. This means better interfaces, well designed screen layouts, printed reports and interaction over networks with databases and electronic records systems. In most cases, you can configure the settings and layout yourself.

### Support worldwide

The Interacoustics name is not only your guarantee of quality and functionality, but also for support. We operate in over 100 countries worldwide through a well coordinated network of distributors and service centres to ensure that you receive total support and service.



#### Other audiometers:

- PA5 Pediatric Audiometer
- AS608/e Screening Audiometer
- AD226 Diagnostic Audiometer
- AC33 Clinical Audiometer
- AC40 Clinical Audiometer
- Equinox<sup>2.0</sup> Clinical Audiometer

## Audiometry and Middle Ear Analyzer in one:

• AA222 Audio Traveller

### Video Otoscope:

VIOTTM



Sales and service in your area:

Read more here: www.interacoustics-us.com/AD629e



Phone: 1-800 947 6334 · Fax: 1-952-903-4200 E-mail: info@interacoustics-us.com 7625 Golden Triangle Drive, Eden Prairie, MN 55344 Web: www.interacoustics-us.com