



DVRemote Installation (Windows XP)


1. Insert an USB into the x-ray computer.
2. Open the [Windows Explorer](#) folder on the computer

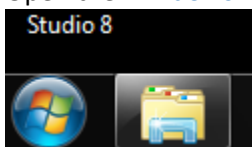


3. On the left-side of the screen click on the [C-drive](#) > [DuoView Installation](#) > [DVRemoteInstallationV2](#)> right-click on [DVRemoteInstallationV2](#) and select [copy](#).

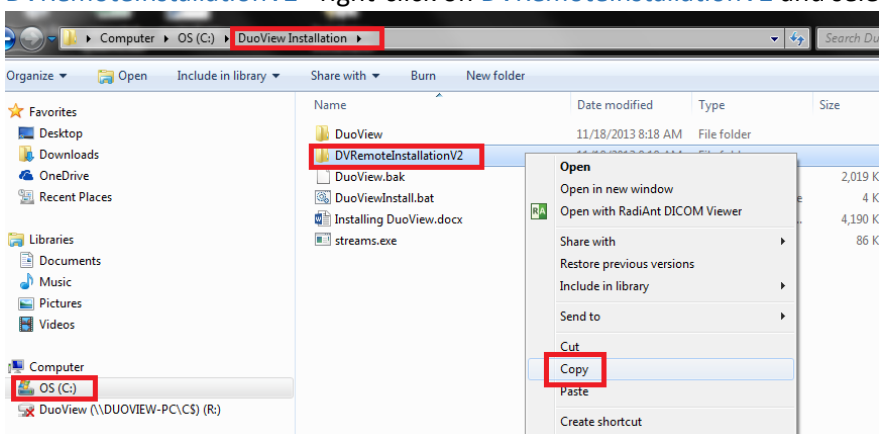

4. Then [paste](#) that folder onto the USB. Remove the USB from the x-ray computer and place it into the computer that you would like to view the x-ray images. Once placed in the computer open the USB and find the [DVRemoteInstallationV2](#) folder and [copy](#) it onto the computers [C-drive](#).
5. Open the [Windows Explorer](#) folder on the customers computer

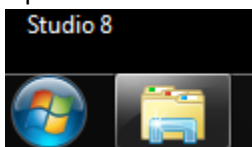


6. On the left-side of the screen click on the [C-drive](#) > [DuoView Installation](#) > [DVRemoteInstallationV2](#)> right-click on [DVRemoteInstallationV2](#) and select [copy](#).

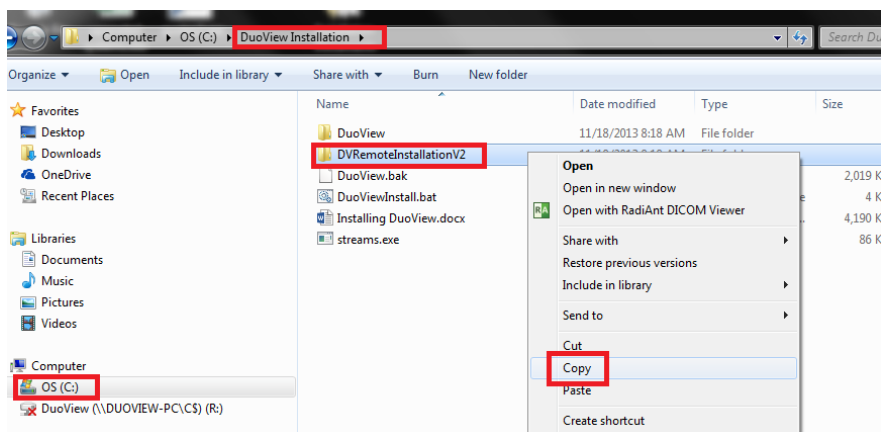

7. Then **paste** that folder onto the computers **C-drive**
8. Double-click and open the folder, and double-click on the file named **ConfigureRemoteDuoView**.


Name	Date modified	Type	Size
addRegKey.cmd	10/14/2013 3:49 PM	Windows Comma...	1 KB
ConfigureRemoteDuoView.bat	10/14/2013 3:55 PM	Windows Batch File	2 KB
DuoView Remote.cmd	3/5/2012 2:49 PM	Windows Comma...	1 KB
Installing Remote DuoView.docx	3/5/2012 6:12 PM	Microsoft Word D...	508 KB
vcredist_x86.exe	3/19/2010 12:40 PM	Application	4,955 KB

9. A black batch file will open- **select 1 and press enter**- two black boxes will appear on customer computer and then disappear, then the **R-drive** will open.

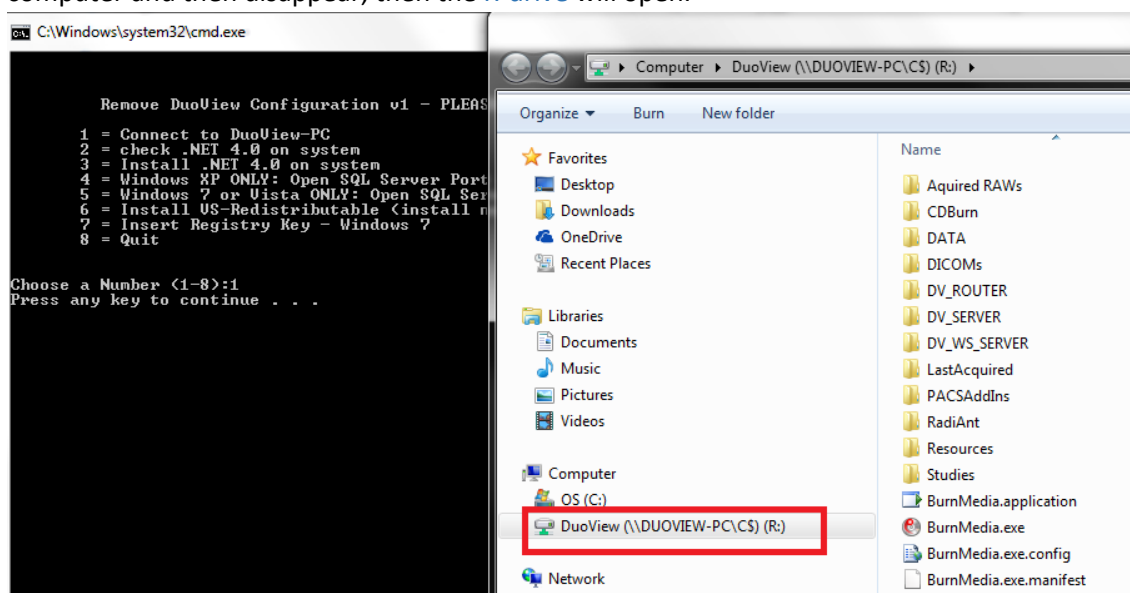

10. Close the **R-drive** screen, go back to the black batch file and **press any key**- then **select #2 and press enter**.

11. Another black window will appear, in this window please make sure that the top line says **v4.0-** if it **does** close that window and go back to batch file.
- If it **does not**, then close the black window and go back to the original batch file and select #3 to download .NET 4.0

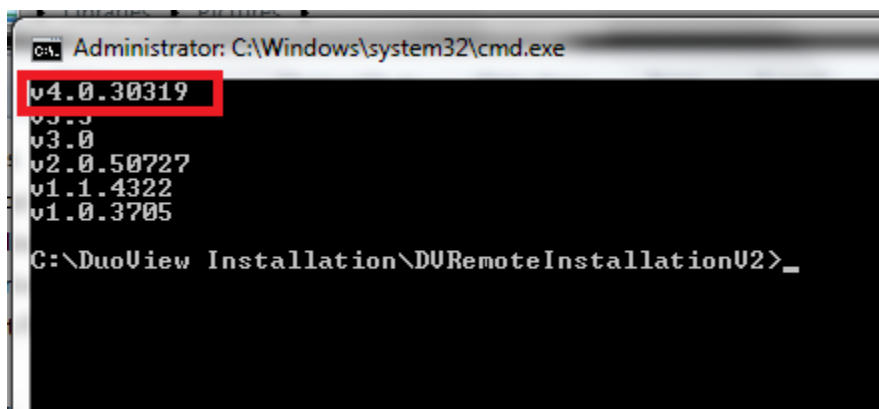

12. Select #4 and press enter- make sure you get an **ok**.
13. Select #6 and press enter-
- If the computer has the correct version of the VS-Redistributable or newer- you are done with this step.
 - If the computer does not have the correct version of VS-Redistributable then press **I** agree to terms and install.
14. Close the batch file.
15. Go back to the **DVRemoteInstallationV2** folder> drag and drop the **DuoViewRemote Icon** onto the desktop.

Name	Date modified	Type	Size
addRegKey.cmd	10/14/2013 3:49 PM	Windows Comma...	1 k
ConfigureRemoteDuoView.bat	10/14/2013 3:55 PM	Windows Batch File	2 k
DuoView Remote.cmd	3/5/2012 2:49 PM	Windows Comma...	1 k
Installing Remote DuoView.docx	3/5/2012 6:12 PM	Microsoft Word D...	508 k
vc_redist_x86.exe	3/19/2010 12:40 PM	Application	4,955 k

16. Double-click to start program- because the program is accessing the database that resides on the main x-ray computer it will be slow. Please be patient, as double-clicking the icon more than once will cause the program to shut itself down.



17. You may be prompted to configure the databases before the software opens- to do so simply click the [Next button](#) until you reach the last page and then [select Finish](#)- once this is complete the software will open automatically. Please be patient.