

UltraClot®

1+ One Step

Z-Fold Hemostatic Gauze with Compression Bandage
+ 2nd mobile pad



- Provides a one step treatment for entry/exit wounds
- Reduces treatment time
- Eliminates need for extra compression bandage
- Prevents rebleeding

Applications

- First Responders
- Military
- Police
- EMS

- FDA K082601
- US Patents 8409629
8609129

Indications for Use: As a sterile hemostatic dressing for emergency external use and temporary wound treatment to achieve hemostasis in cases of moderate to severe bleeding

How it works

1. Gauze is packed into wound
2. Direct pressure is applied
3. Clot forms rapidly
4. Bleeding stops
5. 2nd mobile pad is placed over exit wound
6. Compression bandage is wrapped and locked in place

UltraClot® 1+ One-Step Advantages

- ◆ Stops bleeding rapidly
- ◆ No wound enlargement required
- ◆ Conforms to any wound site
- ◆ Stable clot
- ◆ Effective on heparinized blood
- ◆ Easily removed by standard irrigation & suction
- ◆ No additional debridement required
- ◆ Non-exothermic
- ◆ Effective in any weather
- ◆ 3 year sterile shelf life

4.5 in x 4.1 yds Gauze with 6 in wide
compression bandage + 2nd mobile pad

Emergency Medical Devices

Box #367
Loxahatchee, Florida 33470
emeddevices@bellsouth.net

MADE IN USA

www.emeddevices.com
Tel: 561.793.9773
Fax: 561.795.9971