

ShroudGuard™

OR Table Pedestal Protectors

Installation Instructions

Bracket Installation Suggestions

Installation Hint: Place and install brackets and fence before removing adhesive tabs from Velcro®.

- Plastic Restraint Fences should be located at least 1.5 inches (two finger widths) from the pedestal.
 - Place support brackets on each side of the surgical table base as shown in Figure 1.
 - Remove the plastic backing on the Velcro strips attached to the back of the side bracket and adhere panel to the side of the surgical table base as shown in Figure 1.
 - Remove the remaining plastic backing on the base of the bracket and adhere to the surgical table base as shown.
 - Velcro adhesion will provide some flexing. Permanent two sided adhesive is available.
-
- Remove opaque film from plastic Restraint Fence.
 - Locate and loosen thumb screw sets to allow Restraint Fence to slide into slots.
 - Insert and center Restraint Fence in the slotted vertical holders.
 - Tighten thumb screw sets until snug.



Contents:

Support Brackets - 2 ea.
Restraint Fence - 1 ea.

Hardware may vary from those shown based on OR Table model.

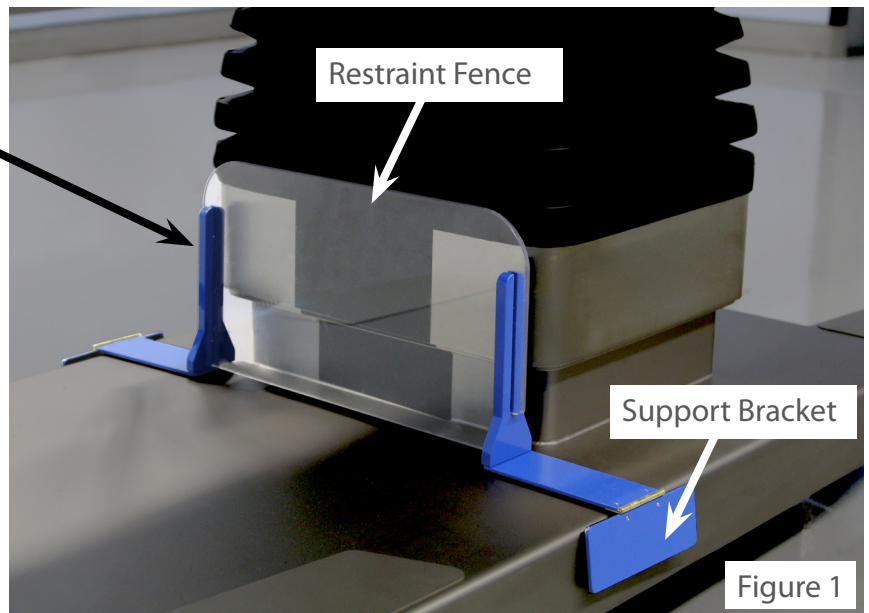


Figure 1