


Peri-jett[®] H-1 Perineal Rinse Device Installation Instructions

Parts included:


- Peri-jett[®] mechanical assembly
- Hanging reservoir bag and hanger
- Sample cleansing solution
- Water supply t-valve connector
- Stainless steel screws (5)
- Stainless steel mounting bracket
- Self-adhesive bumpers (4)

You will need:

- Hand drill with 7/64" bit or hand awl
- Small adjustable wrench or 5/8" wrench
- #2 Phillips screwdriver


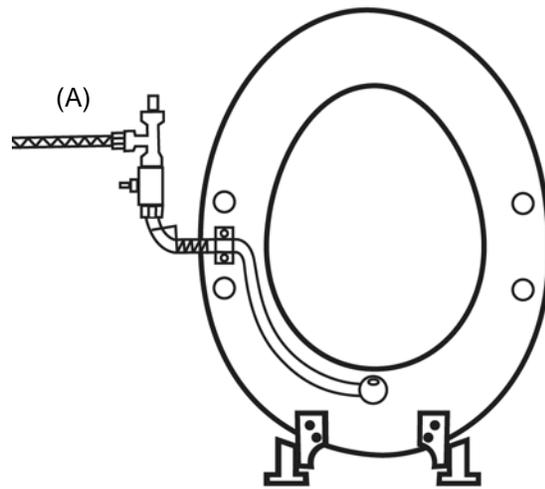


- Remove the water supply line at the closed valve and install the t-valve (B).
- Connect the braided stainless waterline from your Peri-jett[®] and water supply line to the t-valve.

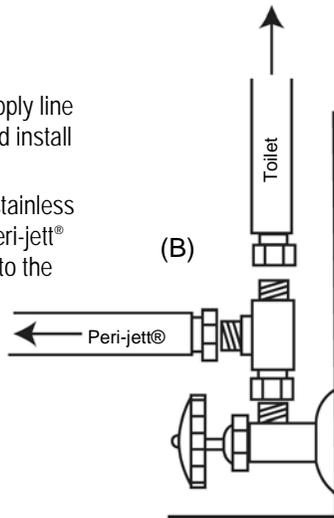

- Close the valve supply water to the toilet tank. Flush toilet to drain the tank.
- Remove the toilet seat and lay it on a flat work surface. This is easier than installation while seat is mounted.
- Center the Peri-jett[®] to the underside of the toilet seat so that the nozzle lies just under the seat opening (1/4" - 1/2" see drawing A).
- Set the mounting bracket over the straight arm of the unit. Mark the wholes with a pencil, drill, and mount the assembly to the seat.
- Once the bracket has been fastened, remove the rearmost screw and use it to anchor the narrow loop of the spring to the mounting bracket. This will tension the spring and keep the Peri-jett[®] against the seat and out of the way until needed.
- Clean the bottom of the seat, remove the adhesive covering and mount the bumpers to both sides of the mounting bracket.

- Lift the toilet tank cover and hang one end of the reservoir hook over the edge leaving the other end free to hang the bag on the outside of the tank.
- Close the friction clamp on the reservoir bag. Open the top of the bag and fill with cleansing solution. Close the top and hang the bag from the reservoir hook.
- Unscrew the cap protecting the Luer fitting on the Peri-jett[®] and connect the fluid line from the reservoir to the stopcock valve on the Peri-jett[®]. Do not discard or remove the protective cap. Simply replace the cap over the Luer fitting when not using the reservoir. (C)
- Open the friction clamp and stopcock valve to allow the cleansing solution to mix with the water in the Peri-jett[®] when the push-button is pressed.

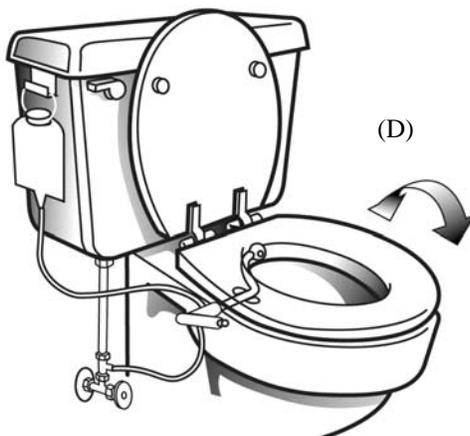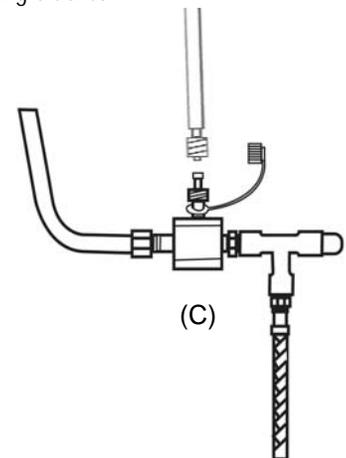

- Move the Peri-jett[®] apparatus back and forth to distribute the solution across the perineal area. The treatment solution may be allowed to stay in place or washed away by closing the stopcock and rinsing with water only. (D)
- Close the stopcock to the treatment reservoir to prevent accidental loss of cleansing solution from the reservoir when not in use.
- Re-open the water supply valve and allow the tank to refill. Your Peri-jett is now ready to use.

Over for cleaning instructions for the Peri-jett[®] and for left handed installation instructions.

LEFT-HANDED Installation of your Peri-jett® H-1

1. Loosen the 5/8" compression nut holding the stainless tubing and rotate ½ turn to the opposite side. Tighten the nut securely.
2. Follow the set mounting instructions for setting the Peri-jett® and mounts by removing the toilet seat and laying it onto a flat surface with the underside up.
3. The front of the toilet seat should be pointing away from you.
4. Mount the Peri-jett® to the right side of the inverted seat.
5. MOST IMPORTANT: Before fastening the spring through the narrow loop and seat fitting, you must unwind the spring ½ turn to allow it to be fixed to the seat using the mounting screw through the rearmost (closest to you) hole in the mounting bracket. This will pull the bidet against the seat if you have done it correctly.
6. Place the seat back onto the toilet and tighten the hardware.
7. The waterline should be run behind the seat and attached to the enclosed t-valve.
8. Turn on the water.

** If the Peri-jett® was not originally ordered for left-handed installation, a waterline extension and coupling may be purchased at nominal charge.

Peri-jett® cleaning instructions

The external surfaces of the Peri-jett® may be cleaned by simply wiping with standard bathroom cleaners used for brightwork on toilets, sinks, and tubs. It is recommended that the surfaces be rinsed with water to remove remaining cleaning agent.

Disinfection of the internal surfaces of the Peri-jett® occurs as the unit applies an antiseptic/antibacterial solution to the user. When the stopcock valve is open, the solution is drawn into the device from the reservoir and exits through the aperture at the end of the stainless wand while the device is being used. The washing solution may be washed away by simply closing the stopcock valve and allowing water to rinse the internal surfaces.

Further internal cleansing is available by removing the reservoir line from the male luer fitting and flushing with a syringe.