


DVRemote Installation (Windows 8)

1. Insert a USB into the x-ray computer.
2. Open the [Windows Explorer](#) folder on the customers computer




3. On the left-side of the screen click on the [C-drive](#) > [DuoView Installation](#) > [DVRemoteInstallationV2](#)> right-click on [DVRemoteInstallationV2](#) and select [copy](#).

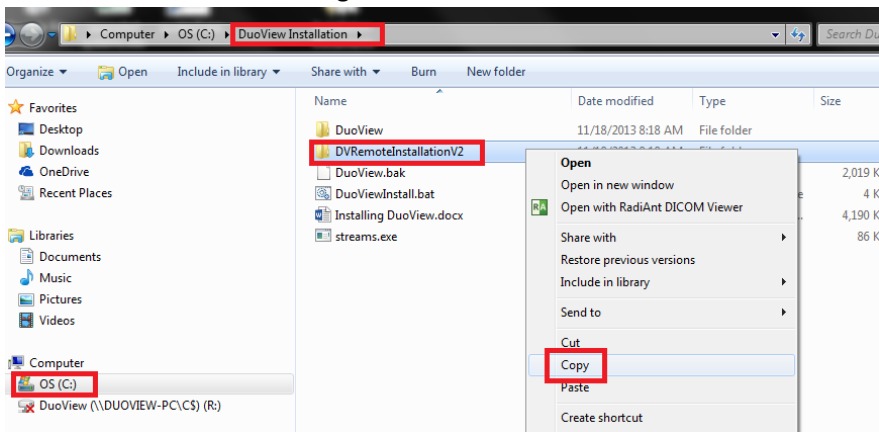


4. Then [paste](#) that folder onto the USB. Remove the USB from the x-ray computer and place it into the computer you would like to view the images on. Once placed in the computer, open the USB and find the [DVRemoteInstallationV2](#) folder and [copy](#) it onto the computer's [C-drive](#).
5. When on the computer's [C-drive](#)- double-click and open the folder, and double-click on the file named [ConfigureRemoteDuoView](#).

Name	Date modified	Type	Size
addRegKey.cmd	10/14/2013 3:49 PM	Windows Comma...	1 KB
ConfigureRemoteDuoView.bat	10/14/2013 3:55 PM	Windows Batch File	2 KB
DuoView Remote.cmd	3/5/2012 2:49 PM	Windows Comma...	1 KB
Installing Remote DuoView.docx	3/5/2012 6:12 PM	Microsoft Word D...	508 KB
vcredist_x86.exe	3/19/2010 12:40 PM	Application	4,955 KB

6. A black batch file will open- [select 1 and press enter](#)- two black boxes will appear on customer computer and then disappear, then the [R-drive](#) will open.

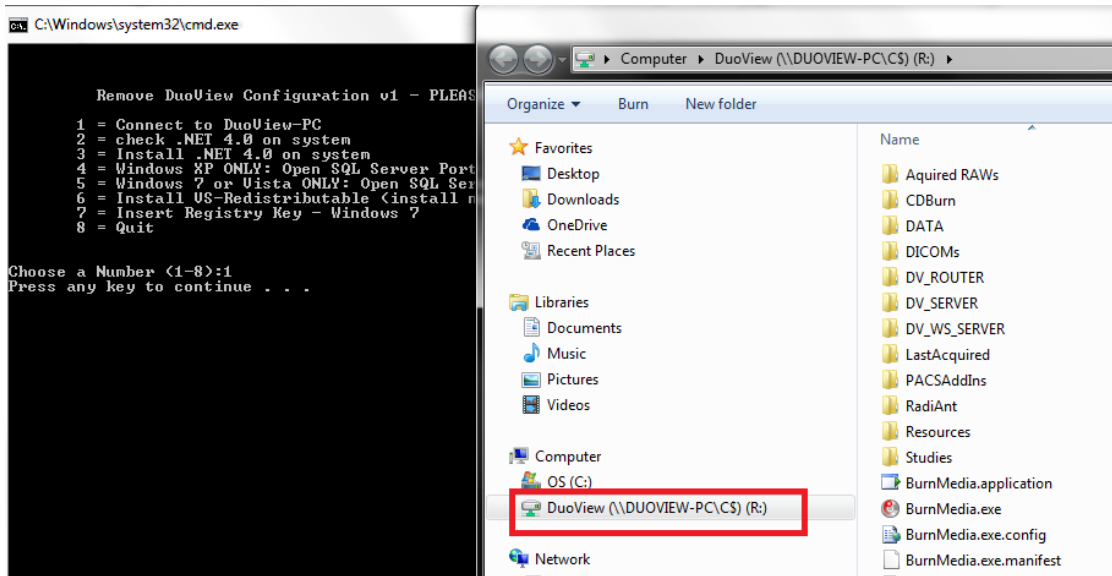

7. Close the R-drive screen, go back to the black batch file and press any key- then select #2 and press enter.
8. Another black window will appear, in this window please make sure that the top line says v4.0- if it **does** close that window and go back to batch file.
 - a. If it **does not**, then close the black window and go back to the original batch file and select #3 to download .NET 4.0- RESTART the computer when download is complete.

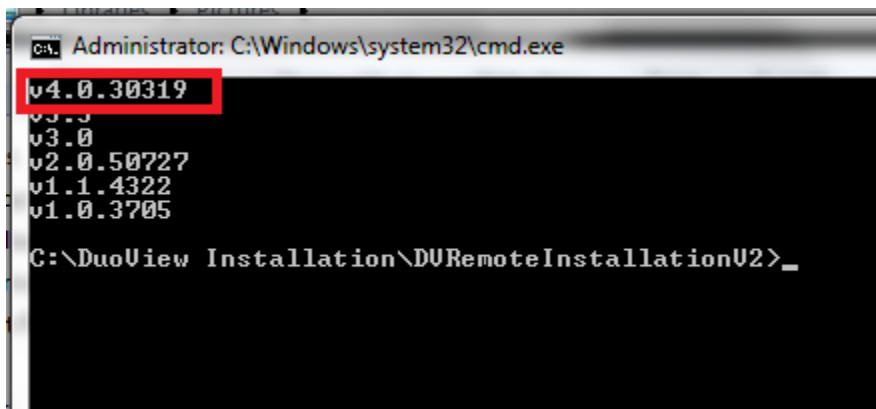


9. Select #4 and press enter- make sure you get an ok.
10. Select #6 and press enter-
 - a. If the computer has the correct version of the VS-Redistributable or newer- you are done with this step.
 - b. If the computer does not have the correct version of VS-Redistributable then press I agree to terms and install.


11. Close the batch file.

12. Go back to the [DVRemoteInstallationV2](#) folder> drag and drop the [DuoViewRemote Icon](#) onto the desktop.


Name	Date modified	Type	Size
addRegKey.cmd	10/14/2013 3:49 PM	Windows Comma...	1 k
ConfigureRemoteDuoView.bat	10/14/2013 3:55 PM	Windows Batch File	2 k
DuoView Remote.cmd	3/5/2012 2:49 PM	Windows Comma...	1 k
Installing Remote DuoView.docx	3/5/2012 6:12 PM	Microsoft Word D...	508 k
vcredist_x86.exe	3/19/2010 12:40 PM	Application	4,955 k

13. Right-click on desktop> select [New > Shortcut > Browse > DuoView Remote > Next > Finish](#).

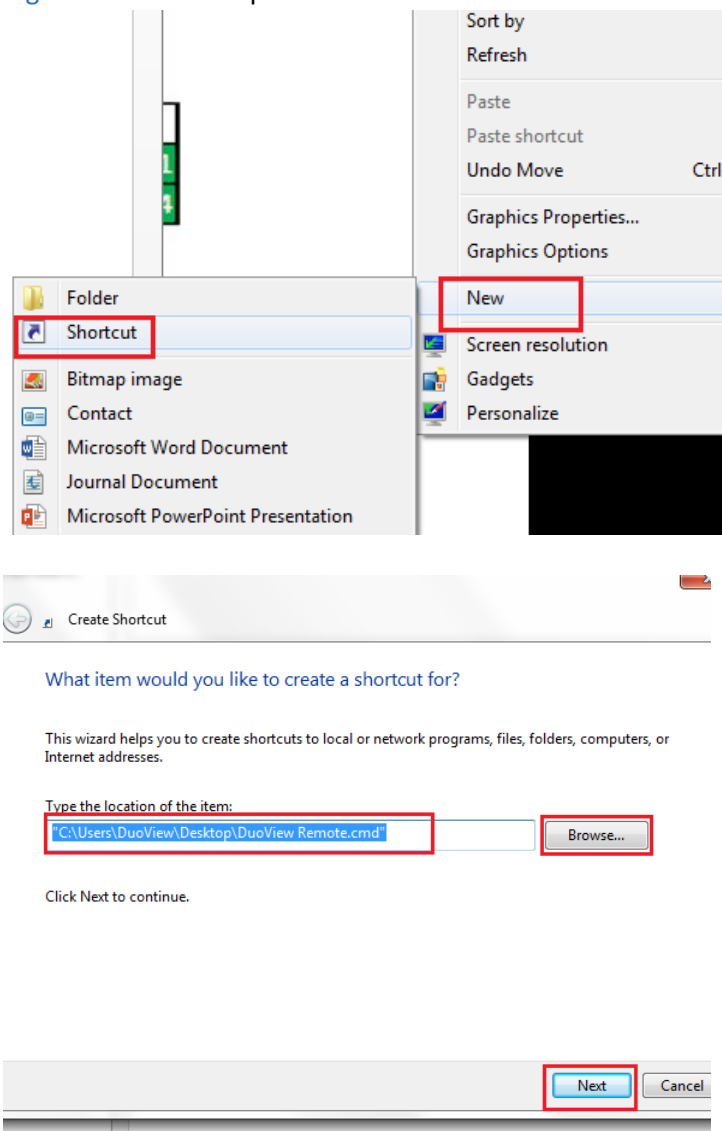


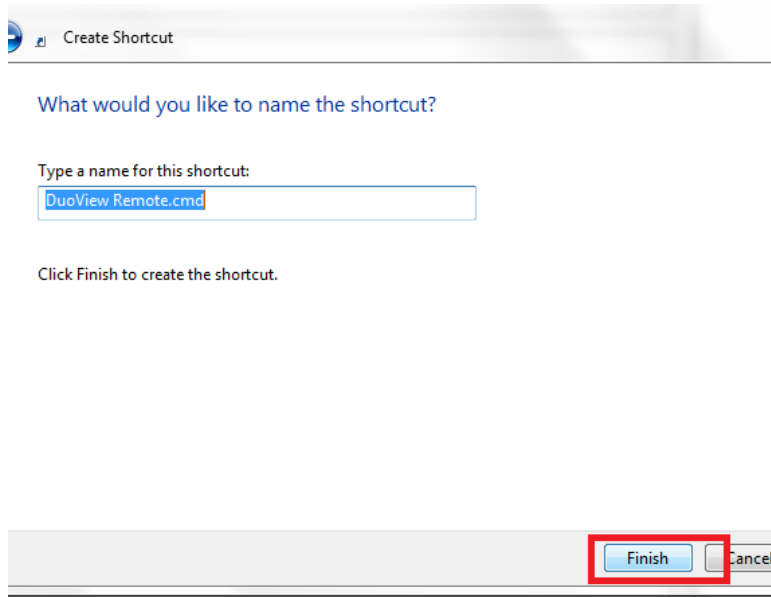

14. Right-click on the new shortcut > Properties > Advanced > Run as Administrator.

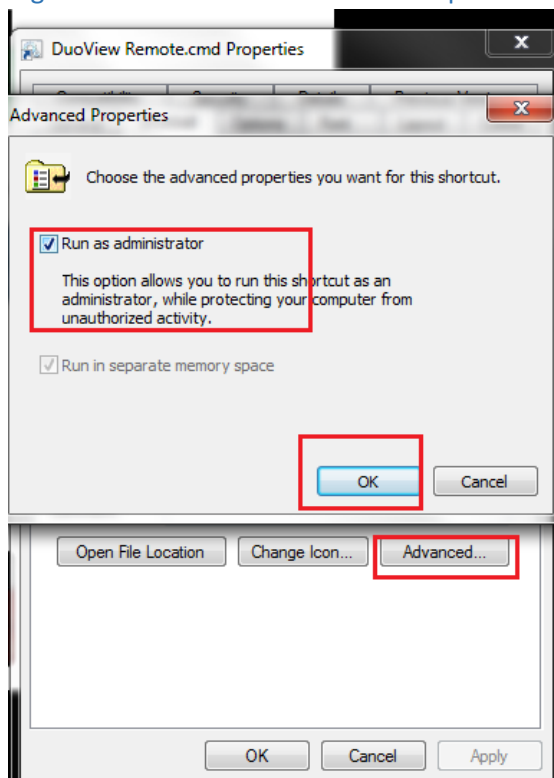

15. Double-click the **NEW SHORTCUT** to start program- because the program is accessing the database that resides on the main x-ray computer it will be slow. Please be patient, as double-clicking the icon more than once will cause the program to shut itself down.

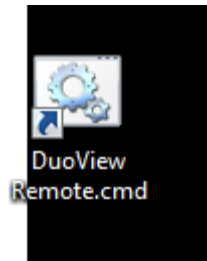

16. You may be prompted to configure the databases before the software opens- to do so simply click the [Next button](#) until you reach the last page and then [select Finish](#)- once this is complete the software will open automatically. Please be patient.