


INDICATIONS

Any pre- or post-operative condition when preventing flexion movement is beneficial. A temporary immobilization device while waiting for physician instruction in emergency situations.


Thoroughly read the following information prior to application.

Product function requires proper application.

Unique Features: 2" or 3" Elastic Compression Straps with contact closure
Velcro® hook "cheater straps"
Also available as a cold therapy product (See figure below and Section E)

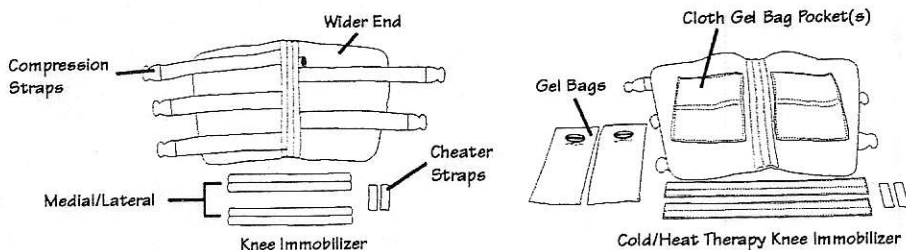**A Position correctly on leg:**

Place immobilizer with posterior stays in the correct position on the leg, foam side to skin. Wider end of the immobilizer positioned toward the proximal position on the leg (Fig. 1).

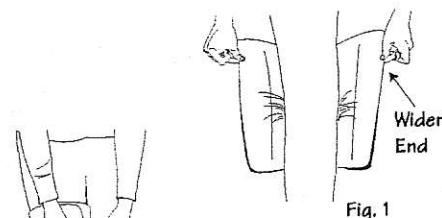**B Close immobilizer:**

Close immobilizer using "cheater" straps (Fig. 2).

Fig. 2

C Place Medial/Lateral stays:

Place medial/lateral stays just posterior of the patella (Fig. 3 and Fig. 4).

Fig. 3

D Secure Compression straps in proper sequence:

Pull compression straps around to fully enclose the leg and secure with Velcro® closure. Close straps in progression as shown in figure 5.

Fig. 4

Note: The length of the immobilizer determines the number of straps on the brace.

E Cold/Heat Therapy Knee Immobilizer:

Insert frozen or heated gel bag into cloth pocket and secure using Velcro® closure (Fig. 6). Then follow the instructions as stated above.

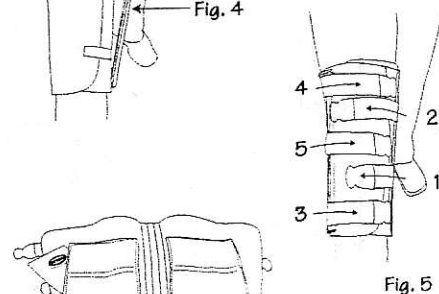


Fig. 5

Note: Please refer to instructions on bag for proper use of gel bag.

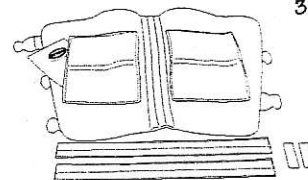


Fig. 6

CARE INSTRUCTIONS

Hand wash using a mild detergent. Rinse thoroughly and air dry completely before applying to the patient.

WARNING: THIS PRODUCT IS TO BE USED ONLY UNDER THE SUPERVISION OF A MEDICAL PROFESSIONAL. NOT INTENDED FOR UNSUPERVISED PUBLIC USE. INCIDENTS OF PAIN, SWELLING, CHANGES IN SENSATION OR UNUSUAL REACTIONS SHOULD BE IMMEDIATELY REPORTED TO YOUR MEDICAL PROFESSIONAL.

CAUTION: FEDERAL LAW (USA) RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTHCARE PROFESSIONAL.

MANUFACTURER MAKES NO GUARANTEES THAT INJURY WILL BE PREVENTED USING THIS PRODUCT.