TORBOT*

CONTINUING CARE

BURNS AND COMPRESSION GARMENTS

Donning Instructions

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Torbot Group, Inc.
Burns and Compression Garments

USA

The garment you have received has been prescribed by your medical advisor;

Following a burn injury more than 80% of third degree burns will develop hypertrophic scarring. The use of compression garments helps to reduce the amount of scarring resulting in a smoother appearance of the skin that is more flexible.

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USA

For Vascular/Lymphedema conditions it has been designed specifically to provide compression therapy, conform anatomically, and allow freedom of movement while providing comfortable and therapeutic compression for the long term management of vascular and lymphedema conditions.

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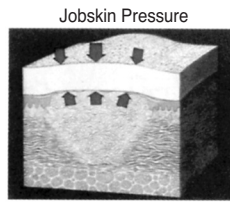
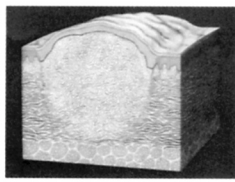
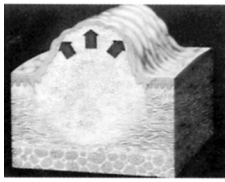
USA

Attention: If you experience severe discomfort, new sores, pain, or swelling after donning your garment - remove the garment immediately and contact your health care provider. Wearing a custom compression garment with open wounds should be discussed with your health care provider. Small unhealed areas or other open wounds should be covered with an appropriate dressing to prevent irritating the wound, and soiling the garment.

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USA

Hypertrophic scarring is an excessive overgrowth of dense collagen tissue, and often red, pink, or purple in appearance at the site of the healed skin defect.

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Hypertrophic scarring comes from skin growing rapidly as it tries to heal.

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The Burn pressure garment replaces the skin pressure which has been lost due to the burn. This pressure is needed to form a flatter, smoother scar.

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Once your body weight has stabilized, you will be measured for a Burn pressure garment that will be made especially for you.

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USA

Vascular/Lymphedema conditions usually cause swelling in the affected limb.

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Once your edema has stabilized, you will be measured for a pressure garment that will be made especially for you.

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USA

Your doctor and therapist will let you know when you should start wearing your pressure garment.

You must wear your Burn pressure garment daily for 24 hours per day for the time period your medical advisor thinks is necessary. For optimal wear time, two garments may be necessary — one to wash, one to wear. Remember to change and wash your supports daily.

Gloves and face mask may be removed while eating.

Remove your garment and launder per these donning instructions. Do NOT wear while bathing.

Garment is an open weave and does not provide protection from sun exposure.

Garments exposed to chlorinated water should be washed per the donning instructions as soon as possible.

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
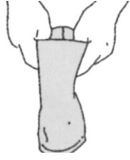
Knee Length Garment Without Zippers


1

2

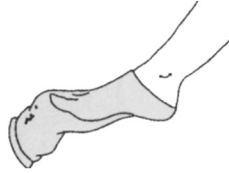3

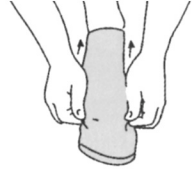USA

Check the label on your pressure garment for left or right. Starting with the garment inside out, tuck the foot portion inside up to the heel and hold it with both hands, seam facing down.

Stretch your garment as wide as possible and in one motion, pull it onto your foot. Put the heel pocket over the back of your heel.

Take hold of the fabric just past your toes always at the sides, never the seams.

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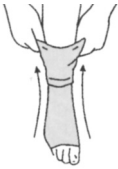
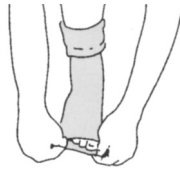
7

USA

Pull it up over your foot past the ankle. It should now be right side out. Adjust the garment so that the heel is in place.

Adjust the toe area for comfort by pulling your garment toward your toes and releasing.

Now work it all the way to the knee — about 2" (5 cm) at a time — smoothing the creases as you go.

For knee length garments, the top band should reach within 1/2" (1.2 cm) of the bend of your knee. Remember, pull on the sides, never on the top or seams.

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
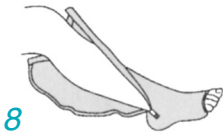
Knee Length Supports With Zippers

Check whether support is for left or right. Open the zipper completely.

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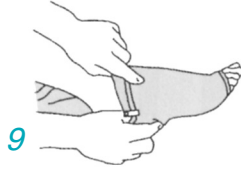
Point your toes and slip your foot into the foot area. Place your thumb between the zipper and your leg to protect your skin.

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Hold the zipper together with one hand approximately 4"-6" (10-15 cm) above the zipper tab.

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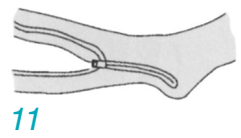Zip it closed, keeping the elastic backing between your skin and the zipper. Adjust any wrinkles or uncomfortable spots by gently pulling the fabric until the pressure is even.

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Thigh Length Garment

Waist Height Maternity Garment

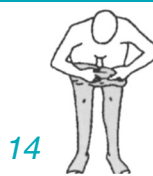
Start with the instructions for knee length (Fig. 1-7). Continue working your support up about 2" (5 cm) at a time.

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
Stretch it evenly and fasten to a garter belt or girdle. If it is uncomfortable behind the knee, ease it down slightly from the knee until the pressure feels more even.

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Start with the instructions for knee length garment (Fig. 1-7). Continue working it up to the middle of your thigh about 5 cm at a time.

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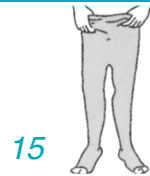Stand up. With both hands, grab a handful of fabric and pull it up to your waist.

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18

USA

Fit it snugly but comfortably over the groin area.

Adjust your pressure garment so that it is snug at your midsection. This will keep it from sliding down. Pull the midsection to its proper height.

Slip your hands, palm out, into the back and lift the fabric up and away from your skin. Let it fall back to fit snugly over buttocks.

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22

USA

(This garment may be difficult to put on alone. You may need help.) Follow the directions for zippered support (Fig. 8-11).

Stand up and continue to close zipper holding the zippered area together 4"-6" (10-15 cm) above the zipper tab. Adjust the leg along the inside over your groin and buttocks.

From this point, use the instructions for waist height supports without zipper (Fig. 14, 15,16) to ease your pressure garment comfortably into place. Adjust garment if necessary.

Follow the steps for putting on supports with zippers. (Fig. 8) Ease your foot through the entire length of the support. Refer to Fig. 15-17 to ease your pressure garment comfortably into place.

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Vest: Sleeves Without Zippers

Always apply the garment on the more severe arm first. Refer to (Fig. 27) for sleeves. When sleeves are fitted into the axillae, close the Velcro brand closure or zipper. Adjust neck edge and waist for comfort.

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Vest: Sleeves With Zippers

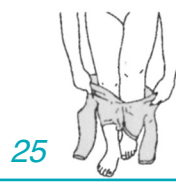
Refer to (Fig. 31-32). Do not close the zippers until the body is fitted. Position the body of the vest. Close the zippers in the sleeves and refer to (Fig. 29) for correct placement of optional pad.

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
Body Suits and Briefs

These styles must be stepped into as body opening runs from neck to just below waist. Adjust legs and lower body portion for comfort.

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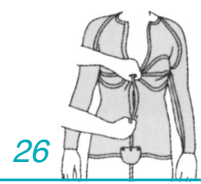
The sleeves are applied as sleeves with and without zippers (Refer to Fig. 27, 31 and 32). Close the body zipper and then the sleeve zippers. Adjust the leg and body for comfort.

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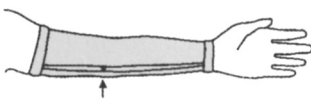Arm Sleeves

27

Slip your hand through the entire sleeve keeping the seam in line with your little finger. Ease the sleeve up your arm. The black dot indicates where your elbow should be. It may be helpful to brace your hand against a wall.

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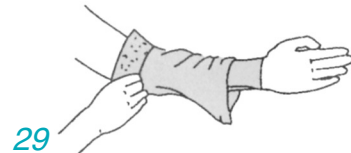
28

If a gauntlet (fingerless glove) is attached, the seam should run from the palm of your hand, then curve over your elbow to your upper arm.

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
29

Optional: Insert pad at bend of elbow for added comfort. Fold sleeve down to elbow. Tuck pad in at elbow for added comfort. Smooth the sleeve up over the pad and continue to ease it up the arm.

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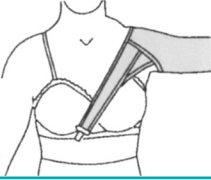32

USA

Straps should be securely fastened.

(Assistance may be necessary). Open zipper fully. Place your fingers inside the sleeve at the "closed" end and stretch it as wide as possible. Slip the sleeve over the arm so that the black dot lies at the elbow. Refer to fig. 29 for correct placement of optional pad.

Close the zipper, holding the zipper together with one hand 4"-6" (10-15 cm) below the zipper tab. Be sure to keep the elastic backing between the skin and zipper. Continue down the arm to the wrist.

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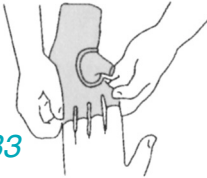
36

USA

Method 1: Turn glove inside out. Fold tips of fingers inside. Place tips of glove on fingers. Work fabric down until thumb tip is covered. Turn hand of glove right side out over fingers.

Method 2: Slip your hand through wrist of glove and into fingers.

Adjust the fingers by carefully pulling on the seams until the glove fits snugly into the web spaces between the fingers. Be sure the fabric is smoothed out evenly over the fingers and hand. Hold the hand up to the light. You should not be able to see light through the web space.

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Chin Strap

37

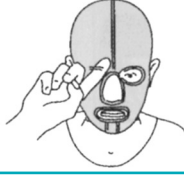
Open Velcro® brand closure. Hold chin in place with one hand, if you have assistance, or press chin against upper sternum. Gather fabric and stretch over head.

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38

Adjust the facial features. The face mask must have the fabric distributed to fit properly. Adjustments may be made by slipping fingers through openings and placing fabric where desired.

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39

Method 1: Follow directions for masks (Fig. 37 and 38).

Method 2: Close the neck portion around patient's neck. Pull head strap and back elastic over head. Refer to Fig. 38 to make adjustments.

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Wear and Care Instructions

Machine wash warm.
Separate light from dark colors.
Wash garments daily.

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Your pressure garment may be dried flat, drip dried or tumble dried on low setting. Avoid direct sunlight and excessive heat.

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Do not use chlorine bleach.
Do not iron.
Do not dry clean.
Use only water base moisturizers and lotions.

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Contraindications

USA

Your support(s) has been constructed from spandex and Nylon.

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Active Cellulitis, Arterial Ischemia, Cutaneous Infection, Diabetic Neuropathy, Arterial Insufficiency.

Infants and children may experience periods of rapid growth. Consult your health care provider if concerned garment is affecting growth, or if growth is affecting garment wearability.

Contraindicaciones:

Celulitis activa, isquemia arterial, infeccion cutanea, neuropatia diabetica, Insuficiencia Arterial.

Custom Warranty

Warranty in general

Torbot Group-Jobskin Division will replace pressure garments for workmanship (see Changes in Measurements below for clarification on fit), free of charge for 30 calendar days (Warranty Period) of being shipped from our Manufacturing facility.

If there is a problem, what happens?

As soon as a problem is identified and communicated to The Torbot Group, another garment will be manufactured at our discretion. If this is the case a replacement will be manufactured immediately. We will not wait until the problem garment is returned (unless the problem cannot be determined without seeing the garment).

Warranty Period Extension

The Warranty Period may be extended at the discretion and prior approval of Torbot Group.

Returning Garments

It is expected that any problem garment be returned to Torbot Group as soon as possible. Please provide as much information as possible so that we can conduct a full investigation. Without the return of the garment, it is difficult to determine the exact nature of the problem and find a remedy.

All returned garments must be clean/laundered to minimize any potential risk of infection. Torbot Group is unable to accept any garment that is not clean when returned and such garments may be destroyed immediately. Freight charges on authorized returns must be prepaid

Changes in Measurements

Please advise Torbot Group of any problems with fit within the Warranty Period.

If the measurement difference is of comparative significance, and/or outside of the Warranty Period, a replacement garment will generally be treated as a reorder. Please note that changes in measurements due to weight gain or loss are not replaced at no charge. Out of Warranty garments are treated on a case-specific basis.

Changes in Options

Should an option be omitted from the original order, Torbot Group will review the documentation and contact you to discuss an appropriate course of action.

In some instances it may not be possible to make the requested adjustment and Torbot Group will contact you to determine how to proceed. If an option is added during the warranty, we will charge only for the option, and not for the adjustment.

Changes in Style

If a new style from an original garment is requested, i.e., thigh high cut to a knee high, Torbot Group will make every effort to accommodate this requirement. However, no credit will be given for difference between the costs of the different styles.

Adjustments

If adjustments are required within the Warranty Period in order to achieve an improved fit, these adjustments will be done at no charge.

Any alterations made to the garment (by parties other than The Torbot Group), will, automatically void the Warranty.

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* *Velcro* is a registered trademark of *Velcro USA, Inc.*