

SLEEP SALES

[855.244.SLEEP \(7533\)](tel:855.244.SLEEP) | [info@sleepsales.com](mailto:info@sleepsales.com)

# SleepView®

## Training Manual


**MULTI-NIGHT EDITION**



**SleepView™ Web Portal**

## SleepView Web-Portal Step-by-Step Instructions

Open your Internet browser and go to: <https://www.SleepSales.com>

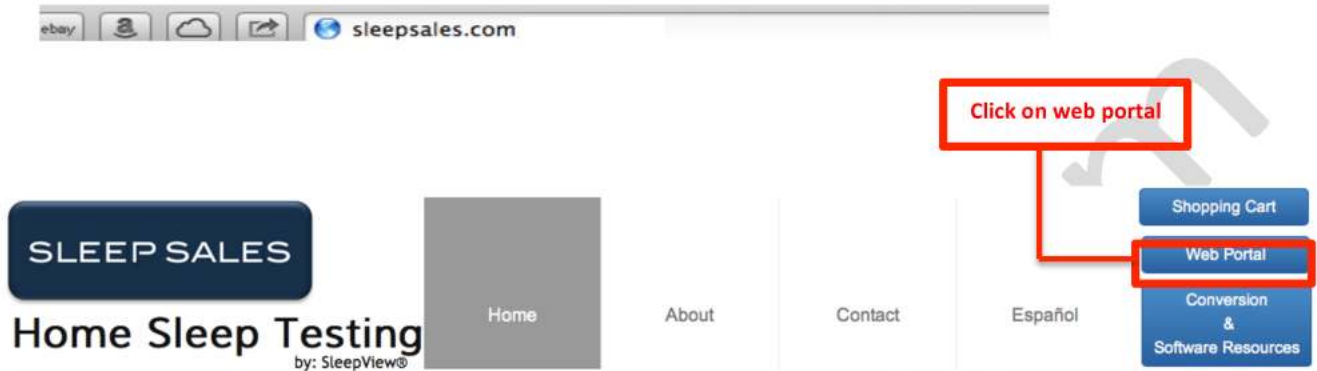

Enter your username and password:



To Schedule a new study, Click on Step 1.




This will take you to the patient information. Fill out all the tabs with the new patient's information




Select Date of Study:



A Month box will appear for you to select the study date:

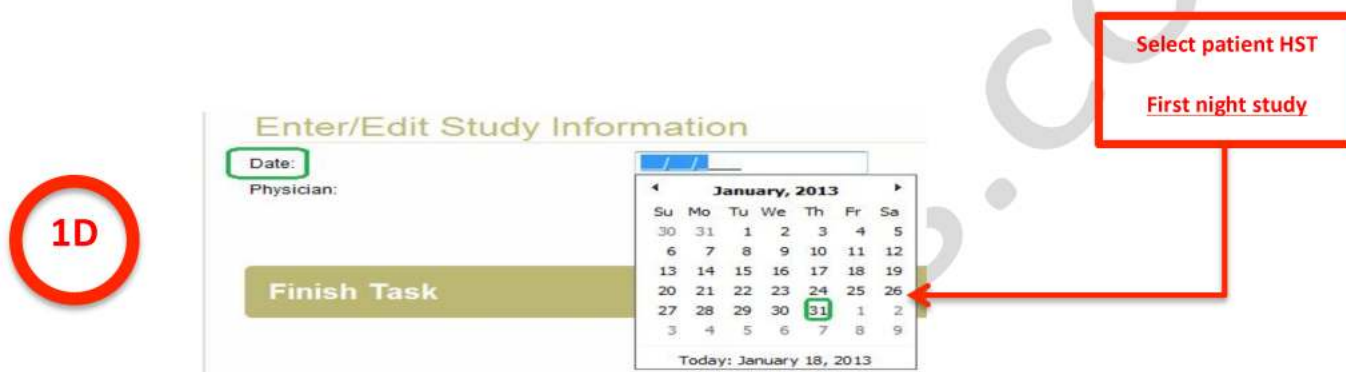

Select Healthcare Provider

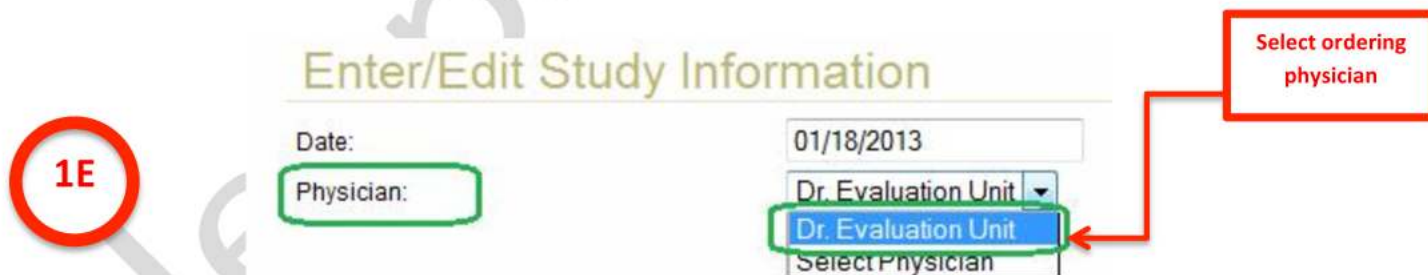

Click: Save Study Info



Select device: Match the last four digits of your SleepView® serial number with the numbers showing on the drop-down list. If your device's serial number is not listed, select "new device". (The device number will appear after 1<sup>st</sup> check in).

1F


## Select Device to Prepare

NOTE: "Prepare Device" will launch SleepView® Manager.

IMPORTANT: Please "Commit Study" after successful preparing the device.

Please select device from drop-down list:

New Device
0##5
0##4
0000

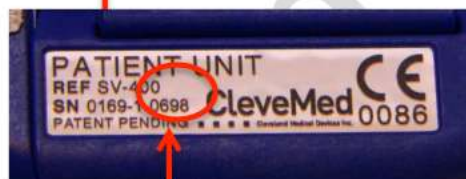

Click prepare device

Prepare Device

A window will pop-up, follow directions are on screen to plug in the SleepView monitor to the computer and turn on

1G

## Connect Device

Connect your SleepView to the computer using the USB cable provided. Next, press the (ON) button and verify that the power LED lights or blinks.


Take the black USB Cord and plug into your computer.



Open the Back of the SleepView monitor to find Micro port to attach the other end of the USB cord



Connect the device and follow the prompts:

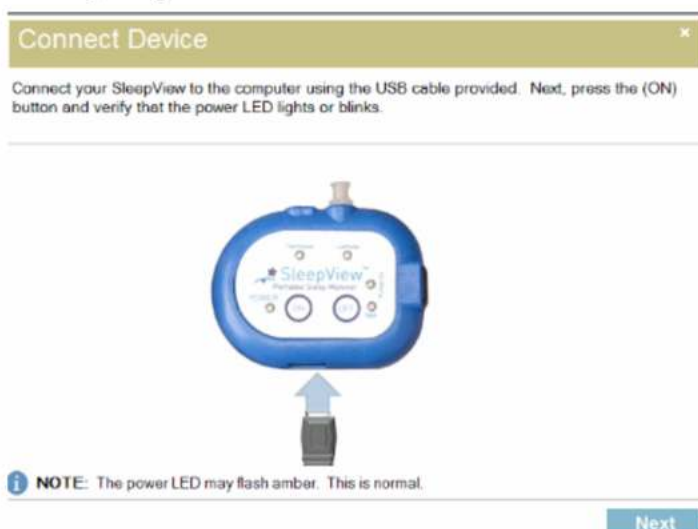

Check "begin recording" box

1H



Boxes to enter time and confirm date will show at this time and must be filled

☒ Begin recording patient at a specified time and duration

1I

**Prepare Device**

Prepare the device for the next study (8400126796) with the options provided below and then click <Next>

☒ Begin recording patient at a specified time and duration

Start Time: 08/22/2013 01:06 PM

Duration: 8 hours 0 minutes

☐ Study will take place in different time zone

**WARNING:** This operation will erase all previously recorded studies from the device.

[Back](#) [Next](#)

Click on numbers to be changed, type in appropriate time.

Month Day Year Hour Min AM/PM

Start Time: 01/18/2013 10:16 AM

Duration: 8 hours 0 minutes

**Duration must be 8 or greater than 8 hours**

**When done, click next**

**WARNING:** This operation will erase all previously recorded studies from the device.

[Back](#) [Next](#)

The software will start programing the device with the information you have set up

**Downloading Parameters**

Your SleepFlow device is being prepared for the next study. Please click "Next" when it becomes available.

Setting device clock:	DONE
Setting auto-start time:	DONE
Erasing device memory	DONE

**Setting device clock:** 1/18/2013 10:20:05 AM

**Setting auto-start time:** 8400121984 | 1/18/2013 10:16 AM

**Erasing device memory**

**This process will take about 30 seconds**

[Next](#)

Follow prompts on screen

1J

**Insert Battery and Check Device**

Disconnect your SleepFlow and insert a new AAA battery. Then, press the (ON) button and verify that the green LED light is green.

[Next](#)

Unplug the black USB cord from the unit by gently pulling it out of the monitor

## Insert Battery and Check Device


Disconnect your SleepView and insert a new AAA battery. Next, press the (ON) button and verify that the power LED lights green.



Take a new SleepView® Testing Kit for each patient




Take the Battery out of the bag

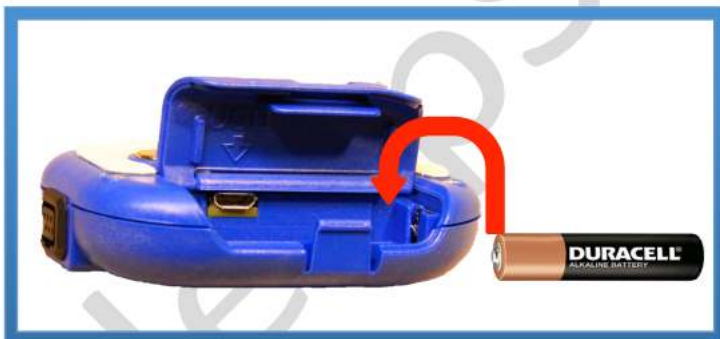


On the back of the SleepView® insert the battery inside the battery compartment



When done, close the back of the device and proceed to press the  button

1K

Turn the SleepView® monitor ON by pressing the  button to verify set up is complete.

Make sure the power light turns green

Press the  Button




If the Power light flashes amber on the SleepView® monitor click here



If the power LED flashes amber, click [here](#).

This will take you back to preparing the device



Step 1  
Prepare Sleepview®  
Monitor for patient's  
home sleep test.  
[Schedule Study](#)

Turn the SleepView® monitor off by holding the  button. Then click on Commit Study

1L



Press the  Button

The SleepView® monitor is programmed for this Patient, click on the “finish”  button followed by “commit study” .

1M



Instructional video: How to Use SleepView Monitor” can be found at:

<https://sleepsales.com/patient-support-videos.html>

SLEEP SALES

Please visit : <https://sleepsales.com/training-manuals.html>

To Download:

SleepView®

Training Manual

MULTI-NIGHT EDITION



V. I



V. II



V. III



V. IV



SleepView™ Web Portal