



Recommended Monitor Mount Placement

The table below shows recommended locations to place the A-dec Ceiling Monitor Mount. Primary recommendations minimize light and monitor arm interference.

IF THE SYSTEM HAS THIS TYPE OF LIGHT:	INSTALL THE MONITOR MOUNT HERE:	
	Primary Recommendation	Secondary Recommendation
Track-Mounted Ceiling Light	C2*	S1 or S2
Fixed-Mount Ceiling Light	C1	S1 or S2
Chair Mount Light	S1 or S2	C1
Preference-Mounted Light	S1 or S2	C1

* If installing at C2, position the monitor mount a minimum of 6-1/2" (165 mm) in front of the track light (towards 12 o'clock).

* NOTE: C2 position restricts comfortable use of monitor to only supine patient position.

A-dec Headquarters
 2601 Crestview Drive Newberg, Oregon 97132 USA
 Tel: 1.800.547.1883 Within USA /Canada
 Fax: 1.503.538.0276 Within USA /Canada
 Tel: +1.503.538.7478 Outside USA /Canada
 www.a-dec.com

**A-dec 511 (Version A)
 Dental Chair
 Mini Template**

Copyright 2019 A-dec Inc.
 All rights reserved.
 86.0905.01 Rev F

A-dec 511 Plumbing Elevations

PLEASE NOTE: This drawing is for reference purposes ONLY. It is NOT TO SCALE.

A-dec Track Light

CAUTION: Please see note below.

NOTE: If the track opposite the transformer is closer than 9-1/2" (241 mm) from a wall, the trolley and post assembly must be installed before the track is installed on the ceiling.