

SENTINEL DEVICE QUICK START GUIDE

Holter patch recording instructions:

Step 1: Attach Holter recorder and patch

1. Snap Holter adaptor to recorder.




2. Attach recorder to patch.



3. Turn patch and recorder over on table with backing facing up.
4. Wipe skin with alcohol wipe and pat dry before applying patch.
5. Peel backing down 2 inches and stick to center of chest, just below collarbone.



6. Press down on patch to stick to chest.
7. Gently pull backing down and out to avoid wrinkling. Press to skin as you go, until patch is stuck to chest.

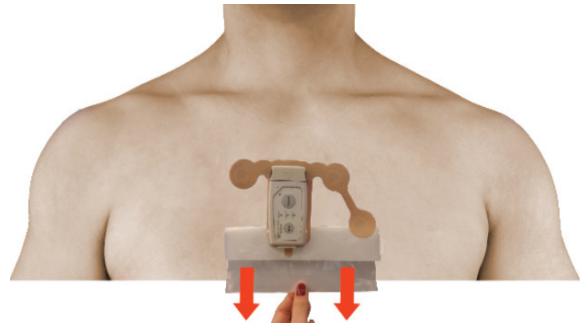

Step 2: Start Holter recording

1. Press and hold Record button until you hear a beep.
2. Release button. Recording is started.

Step 3: End recording

1. When recording ends, gently remove patch starting at top and slowly pulling down to remove recorder and patch.
2. Return patch and recorder to study site.